

Product Specifications (BA06040)

| | |
|-----------------|-------|
| Seat Width | 55cm |
| Seat Height | 50cm |
| Max User Weight | 113kg |

This shower chair has an adjustable base frame to fit over the bath for range 737mm to 787mm.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact CareCo.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened.
- Take particular care when using the product on damp surfaces.



Assembly and Maintenance:

1. Place seat upside down on a flat surface.
2. Insert the bath rests into the tubes underneath the seat - these can be adjusted to one of 3 settings to suit the width of your bath. To adjust remove the bath rest adjustment pins and push the bath rests into the tubes until they are level with the required hole. Once in place push the adjustment pin back into the hole. (Please see the following page for reference)
3. Place the seat over the bath and into place, ensuring the bath rests on both sides are far enough away from the edge of the bath to make them secure.

4. Secure the seat into position by turning the locking screw on each side until it comes into contact with the side of the bath.
5. The seat rotates up to 360 degrees by pulling the lever upwards, to lock the seat in place, release the lever.
6. Ensure all screws, nuts and bolts are periodically tightened.
7. Clean the chair after each use and disinfect periodically.

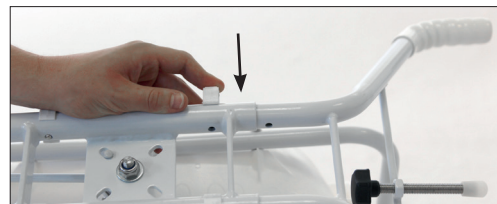
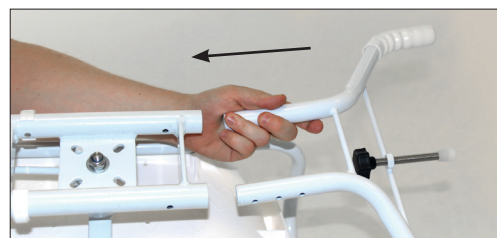


Image reference for point 2:

Warranty:

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement.

If you have any queries please contact CareCo.



CareCo, Hubert Road, Brentwood,
Essex CM14 4JE

www.careco.co.uk