
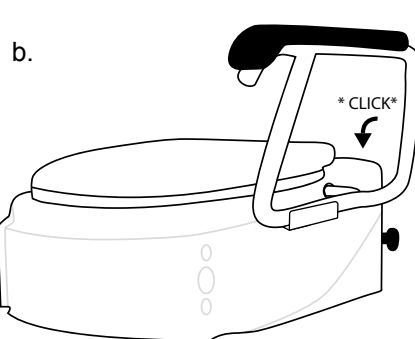


## Before Use:

Check the product for any visible damage, if you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact CareCo for support.

## How to fit:

1. Remove product from carton and read all instructions carefully before attempting to install product.
2. Attach the arm rests:
  - Insert the arm rest tube into the hole in the base of the toilet seat.
  - Line up the arm rest with its bracket on the toilet seat base and push down to 'click' into place.
  - Repeat on both sides.

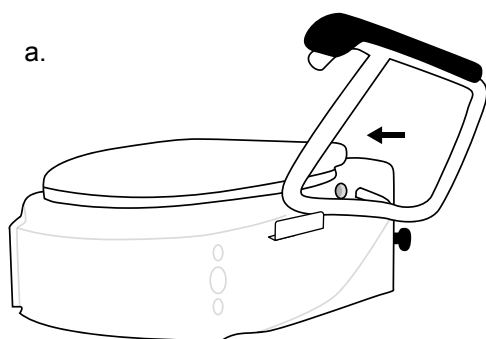


### Specifications:

Seat width: 580mm  
Seat depth: 460mm  
Seat height: 380mm  
Max user weight: 136kg

## 3. Install to your toilet:

- Before installation, please remove your current toilet seat and lid from the bowl.
- Also remove the plastic white nuts from the large bolts on the underneath of the Europa Toilet Seat.
- Line the Europa Toilet Seat up with your existing toilet bowl and thread the screws through the holes on the back of the bowl.
- Screw the plastic white nuts onto the bolts to secure the toilet seat in place.

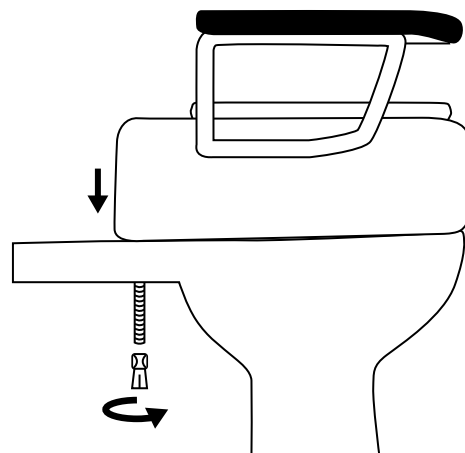

4. Your Europa Toilet Seat is now ready to use and should be cleaned regularly as you would a normal toilet seat for hygiene reasons.

## Warnings:

Do not rock back and forth or sideways when using the toilet seat as this could render the seat unstable, which may cause injury.

## Warranty:

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products. In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement. If you have any queries please contact CareCo.