



BD

Helping all people
live healthy lives

BD Safety-Lok™

Sliding Sleeve Syringe

How to Use the BD Safety-Lok™ Syringe

1. Attachment

Attach the needle on to the BD Safety-Lok™ syringe with a push and a clockwise twist. If pre-attached, tighten the needle to the syringe prior to use.



2. Aspiration

During aspiration, press index finger against flanges to prevent sleeve movement.



3. Injection

Administer medication in accordance with established protocol, make sure that all medication has been dispensed.



4. Activation

Fully retract needle into sleeve until it locks in the protected position. When the "safe lock" indicator green band fully covers the red band and an audible click is heard, the sleeve is locked into position.



Discard immediately in the nearest sharps container.

Don't Forget...

Aseptic technique and proper skin preparation of the site are essential. Follow your institution's policy for safe disposal of all medical waste.

The BD Safety-Lok™ Syringe was designed for reliable performance.

Prior to Activation

Bold Scale Markings

Facilitate accurate dose measurement

BD PrecisionGlide™ Needle Technology


Designed for patient comfort

Large Flanges and Tapered Plunger Rod

Increase control for easier aspiration and injection

Clear Barrel and Sleeve

Make syringe contents easier to see

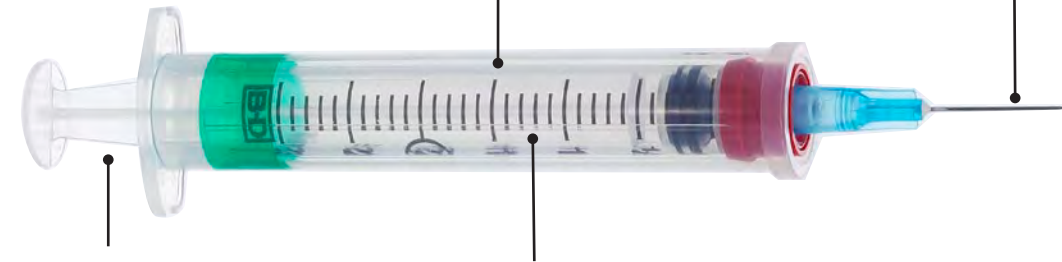

After Activation



Visual Lock Indicator

Confirms when green band fully covers red collar

First in Safety

For more than a century, BD has led the way with new products and technologies, especially in healthcare worker safety.

Nearly 25 years ago, BD pioneered safety injection with the introduction of the BD Safety-Lok™ Syringe.

BD PrecisionGlide™ Needle Technology

- Designed for patient comfort
- Leading brand of hypodermic needle technology in the U.S

Proven Dependability

BD Safety-Lok™ syringe offers you proven performance and dependability.

- Needle retracts and locks into protective sleeve with one smooth motion
- Clear barrel and sleeve allow for dosing visualization
- Large flanges and tapered plunger rod increase control for easier aspiration and injection
- Available in a wide array of sizes – 1mL, 3mL, 5mL, 10mL

Visual Lock Indicator

BD Safety-Lok™ syringe offers clear visual and audible confirmation of the lock position.

- Provides color-coded visual confirmation of the safety lock.
- Lock is confirmed when green band fully covers red collar.



Complete Lock



Incomplete Lock

1ml Insulin Syringe with Permanently Attached Needle

Catalog #	Needle	Packaging
329464	29G ½" U-100 BD Ultra-Fine™	100/Box – 500/Case

1ml Tuberculin Syringe with Permanently Attached Needle

305553	27G x ½"	100/Box – 500/Case
305554	25G x ⅝"	100/Box – 500/Case

3mL Syringe With Detachable Needle

309592	25G x ⅝"	100/Box – 800/Case
309594	23G x 1"	100/Box – 800/Case
309593	22G x 1 ½"	100/Box – 800/Case
309596	22G x 1"	100/Box – 800/Case
309595	21G x 1 ½"	100/Box – 800/Case

5mL Syringe With Detachable Needle

305561	21G x 1 ½"	50/Box – 400/Case
--------	------------	-------------------

10mL Syringe With Detachable Needle

305564	21G x 1 ½"	50/Box – 400/Case
--------	------------	-------------------

Syringes Only, BD Luer-Lok™ Tip

309606	3 ml Syringe Only	100/Box – 800/Case
305558	5 ml Syringe Only	50/Box – 400/Case
305559	10 ml Syringe Only	50/Box – 400/Case

REMEMBER:

Safety-Engineered Devices are required in your Custom and Standard Procedure Trays.

For more information call
1.888.237.2762



BD Medical
1 Becton Drive
Franklin Lakes, NJ 07417
www.bd.com

BD, BD logo, and all other trademarks are property of Becton, Dickinson and Company.

© 2013 BD.

BD 1/13-MSS0075