Multi-Lancet™ Device 2

Lancing Device

Instructions for Use

Please read these instructions carefully before use. Retain this instruction leaflet for future reference.

Use this product only under the direction of your physician or qualified medical personnel.

Intended Use

The Multi-Lancet™ Device 2 is a reusable lancing device used to obtain a capillary blood sample from the fingertip or alternate sites for glucose monitoring or other tests that require blood. The Multi-Lancet™ Device 2 is intended for single person use only.


For Your Safety

NOTE

- If the lancing device is being operated by a second person who is providing assistance to the user, the lancing device should be decontaminated prior to use by the second person.
 - The lancing device is for use by a single person only. It must not be used to collect blood from more than one person as this poses an infection risk^{1,2}.
 - Do not share the lancing device with anyone including other family members! Do not use on multiple patients!
 - User should wash hands thoroughly with soap and water after handling the lancing device.
 - All parts of the kit are considered biohazardous and can potentially transmit infectious diseases, even after you have performed cleaning and disinfection.
- 1 “FDA Public Health Notification: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication” (2010)
<http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>
- 2 “CDC Clinical Reminder: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens” (2010)
<http://www.cdc.gov/injectionsafety/Fingerstick-DevicesBMG.html>

Features and Device Configuration

The Multi-Lancet™ Device 2 can draw blood from a finger and a trace amount of blood from other sites (mainly from the forearm).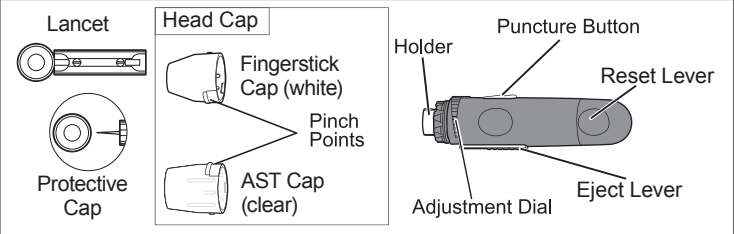

Head Caps

- There are two head caps; one for fingerstick and one for alternate site testing (AST).
- Change the cap to match the blood collection method.
- To remove the head cap, push while holding the pinch points.

Adjusting Lancing Depth

The Multi-Lancet™ Device 2 can adjust the lance depth in 7 levels, as needed. When using the Multi-Lancet™ Device 2 for the first time, select level 3 for finger samples and level 7 for palm, forearm and upper arm samples. If you have soft finger skin, start with level 2 and adjust depth as needed.

- If blood sample is excessive, turn the dial to a lower setting.
- If blood sample is insufficient, turn the dial to a higher setting.


CAUTION

Do not turn the dial beyond 1 or 7 as damage to device may occur.

Resetting the Internal Trigger

If the puncture button is inadvertently pressed before lancing a site, pull the reset lever in the direction of the arrow until a click is heard. This locks the internal trigger. Then take your hand off the reset lever.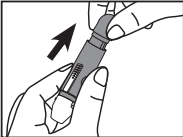


CAUTION

Do NOT pull the reset lever while touching the puncture button and the eject lever.


How to Use

Step 1: Remove the head cap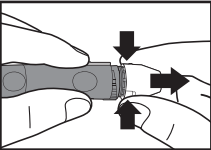

Remove the head cap by pinching and pulling the cap off the lancing device.

Step 2: Insert the lancet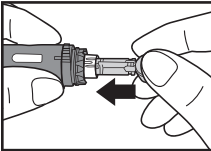


Insert the lancet by pushing down firmly until lancet is fully seated.

NOTE


Do not use if protective cap has been removed.

Step 3: Remove the protective cap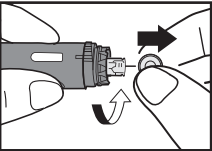


Twist the protective cap off the lancet.

Step 4: Attach the head cap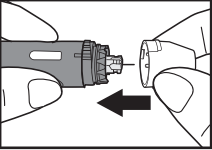

Attach the head cap by pushing it onto the lancing device.

Step 5: Adjust puncture setting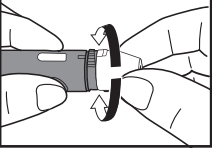


Adjust the puncture setting for the preferred puncture depth level (adjustable depths).

Step 6: Obtain blood sample

NOTE


Consult your blood glucose meter user manual for instructions on how to obtain a blood sample.

Step 7: Attach the protective cap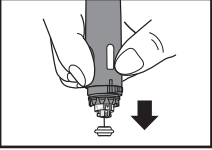

After obtaining your blood glucose results, remove the head cap, then stab the protective cap with the lancet.

Step 8: Dispose of the lancet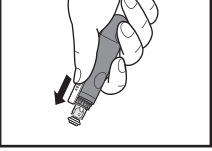

Eject the used lancet using the eject lever and dispose of it in a safe manner to avoid accidental lancet stick injuries.

Cleaning and Disinfecting Lancing Device

Clean and disinfect the Multi-Lancet™ Device 2 with Clorox® Germicidal Wipes, EPA Registration number 67619-12. Other brand wipes with EPA Registration number 67619-12 may also be used.

Purchase Information for Clorox Healthcare® Bleach Germicidal Wipes

Clorox® wipes can be ordered from these national distributors and on the websites listed below.

- Medline..... 800.633.5463
- DNR Supplies..... 763.420.7825
- www.amazon.com
- www.walmart.com


If you need assistance or have questions about cleaning and disinfecting, please contact customer service at 800.566.8558.

NOTE

- Only Clorox® wipes or EPA 67619-12 should be used to clean and disinfect the lancing device.
- Keep Clorox® wipes out of reach of children.

Cleaning and Disinfecting Lancing Device

1. Inspect for blood, debris, dust, or lint on the lancing device.
2. To clean the lancing device and head cap, blood/body fluids must be thoroughly cleaned from the surface of the lancing device and head cap before disinfecting. Dispense the wipe (Clorox®) from its package. **CLEAN** by gently wiping the outside of the device. Dry thoroughly with a dry cloth.
3. **DISINFECT** by gently wiping the outside of the device. Allow the device to remain wet with the disinfectant solution for 1 minute. Dry the device with a dry cloth.
4. Wash hands with soap and water.

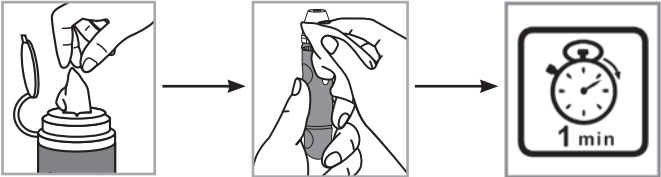Cleaning and disinfection frequency: ARKRAY has validated a 4-year use life by exposing the lancing device to 832 wipes (416) cycles with the disinfectant product listed above. The lancing device should be CLEANED and DISINFECTED a minimum of once per week and immediately when blood is visible, for up to 2 cleaning and disinfection cycles per week over the 4-year use life.

Distributed by
arkray USA, INC.
Minneapolis, MN 55439 USA
www.arkrayusa.com

For questions/comments, contact Customer Service at 800.566.8558, 24 hours a day, 7 days a week, US and Canada.
Made in Philippines

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1 “FDA Public Health Notification: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication” (2010) <http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>

2 “CDC Clinical Reminder: Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens” (2010) <http://www.cdc.gov/injectionsafety/Fingerstick-DevicesBMG.html>

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