

vive®




COMPACT BED RAIL

Owner's Manual
LVA2009

vivehealth.com

WHAT'S IN THE PACKAGE

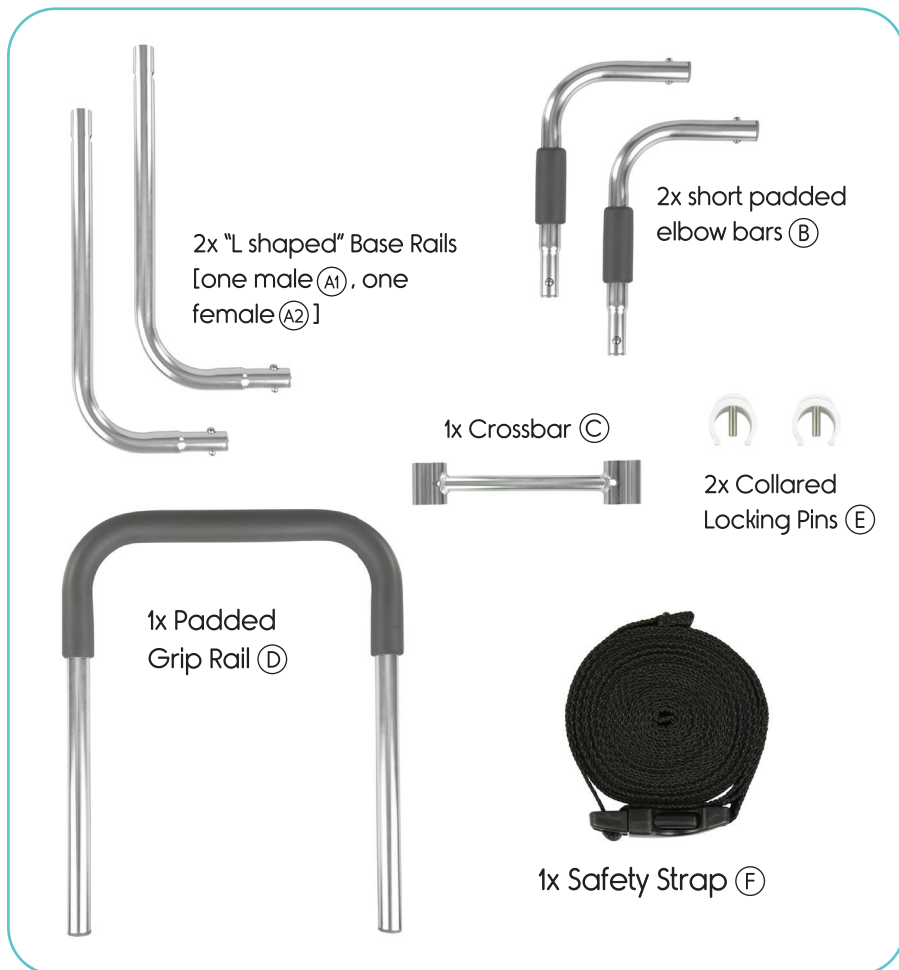


OVERVIEW

Your Compact Bed Rail is designed to provide additional assistance for getting into or out of bed following surgery or other injury. It fits underneath your existing mattress, regardless of size.

ASSEMBLY AND INSTALLATION INSTRUCTIONS

Once installed, you won't really have to worry about the Bed Rail much, as it's built to last in place for a long time. So let's get started installing it:

1. Fit the two (2) long Base Rails together by pushing in the pins on the short end of the male piece (A1) and inserting it into the short end on the female piece (A2) until the pins click into place through the fitting holes. This will create the base for the Bed Rail that will fit under the mattress.

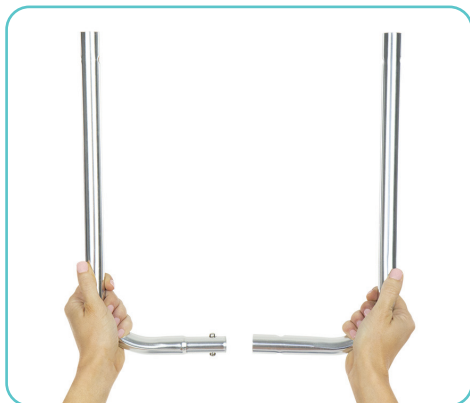


2. Insert the narrow ends of the short Elbow Rails (B) into the open ends of the Base Rails (A1), (A2) by pushing in the pins on each side and fitting them together until the pins click into place. The unpadded half of the Elbow Rails should now be curving up.



3. Slide the inner openings on the Crossbar (C) over the upturned ends of the two Elbow Rails (B) so it is sitting on the spring locking pins. Push one set of pins in and move that side of the crossbar down to cover them. Repeat on the other side, pushing the entire Crossbar down until the pins on both sides pop into place.



4. Slide the Padded Grip Rail (D) into the outer openings on the Crossbar (C), and the Rail should now be curving up.

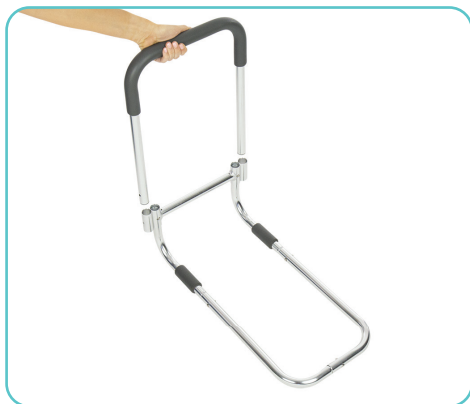

5. Line up the pin holes on the Support Rail (D) with the pin holes on the Crossbar (C), and lock the Rail into place by pushing the Collared Locking Pins (E) into the holes on each side. They should snap right over the Grip Rail and secure the Rail and the base together.



6. Open the buckle on the Safety Strap (F) and tightly loop it around the end of the Base Rail (A1) + (A2) so the middle of the strap is positioned around the base. Position the base of the Bed Rail on the side of your bed under your mattress, and feed the ends of the Strap (with the buckles) all the way under the mattress to the other side of the bed.

7. Find a place to loop it on the other side (usually the bed frame). Fasten the buckle and tighten the Strap as needed. Be sure to hide any excess strap out of the way to avoid a tripping hazard.



MAINTENANCE

The only maintenance required for your Compact Bed Rail is some light cleaning with a damp cloth at regular intervals. After cleaning, make sure that the Bed Rail is completely dry before using the Rail again.

Periodically check to make sure that the Safety Strap is secured and all pins are properly locked into place before use.

If you have any problems with your Bed Rail, including installation, use, or maintenance, please contact service@vivehealth.com.

FREQUENTLY ASKED QUESTIONS

Q: How much weight will the Rail hold?

A: When properly secured to the bed frame, the rail will support up to 300 lbs.

Q: How tall is the Rail?

A: When the Rail is fully raised, it is 20 inches in height.

Q: Can the Rail height be adjusted?

A: No, but by removing the Locking Pin Collars, you can lower the Rail all the way down and out of the way when not in use.

Q: How long is the Safety Strap?

A: The Safety Strap is 157 inches long, making it suitable for up to a king sized bed.

WARRANTY INFORMATION



VIVE GUARANTEE

You are protected by Vive Health's industry leading guarantee and customer service.

QUESTIONS?

Our friendly customer service team is standing by to assist you. Feel free to connect on the phone, email or chat.



service@vivehealth.com



1-800-487-3808



vivehealth.com