

How to use your nebulizer compressor system


If you have been prescribed liquid medication for inhalation, nebulizers are used to change the liquid into a fine mist which allows it to reach deep into the lungs where it can be most effective.

Getting ready


- 1 Put the unit on a firm, flat surface



- 2 Plug the power cord into an electrical outlet.

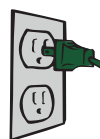

- 3 Connect one end of the tubing to the bottom of the cup and the other end to the air outlet on the unit.

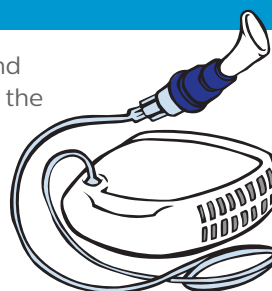


Taking your medication


- 4 Assemble your nebulizer cup and add your prescribed medicine.




- 5 Turn the power switch on "I" to begin your treatment.

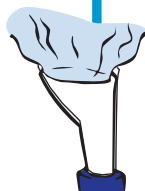

- 6 When taking a treatment, sit upright and relaxed.

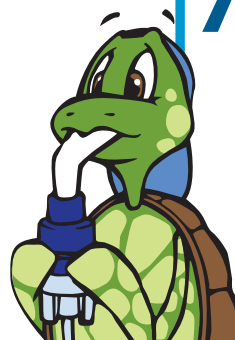

- 7 When your treatment is complete, turn the power switch off "O" and unplug the unit.

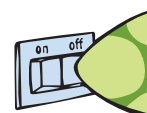

Cleaning your compressor

- 1 Unplug the unit before cleaning.
- 2 Check the filter at least once each week, and replace when dirty.



- 3 Use a damp cloth to wipe down the outside of the compressor at least once each month.



Cleaning your nebulizer

- 1 After each use, take apart your SideStream top, medicine cup, baffle and mouthpiece.
- 2 Wash in warm soapy water.
- 3 Rinse in clean, cold water. Shake out excess water and allow to air dry.

If you have a reusable nebulizer (the light blue one) – Once a week, boil all parts (not the tubing or facemask) in water with 2-3 drops of dishwashing liquid for 10 minutes.

