

QUICK START GUIDE

AYRE™ WIRELESS RAPID-RECOVERY COMPRESSION BOOTS



Ayre™ Boots

- A** 420D Nylon Boots
- B** Hook and Loop
- C** Zipper
- D** C1, C2, C3, C4 (Chambers)



Ayre™ Control Unit

- 1** Power On/Off button
- 2** Lithium Ion Battery
- 3** Global Power Plug
- 4** SET button

INTENDED USE

The Compex® Ayre™ is used to aid:

- Sore or aching muscles from exercise or recreational activities helping you to be ready to go at full speed the next day.
- Sore or aching muscles, pain and swelling of the muscles, vein problems, diabetics blood flow.

This product is a non-medical device and is not used in medical conditions or medical environments.

CONTRAINDICATIONS

This device must not be used for the following conditions:

- Persons with suspected, active or untreated: deep vein thrombosis, ischemic vascular disease, severe arteriosclerosis, pulmonary edema, severe congestive heart failure, thrombophlebitis or an active infection.
- On the legs where cuffs would interfere with the following conditions: vein ligation, gangrene, dermatitis, open wounds, a recent skin graft, massive edema or extreme deformity of the leg.
- Not for use on any neuropathy. Do not use on extremities that are insensitive to pain.
- Not for use where increased venous or lymphatic return is undesirable.

WARNINGS

- Do not attempt to repair the device.
- Do not attempt to open or remove covers.
- Do not remove the pump unit from the cuff.
- Do not attempt to modify or change the device.
- NEVER attempt any service while the device is in use.
- Do not operate in a wet environment.
- Do not immerse in any liquid for any reason. For cleaning and disinfecting instructions refer to "Clear and Disinfecting section".
- Do not place the device in autoclave for any reason.
- Not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- If exposed to temperatures below 10^o(50^oF) allow the device to warm up to room temperature.
- Do not subject the device to extreme shocks, such as dropping the pump.
- Portable and mobile Radio Frequency Communication Equipment can be affected by this device.

PRECAUTIONS

- Stop using the device if swelling, skin irritation or any other unpleasant or painful sensation occurs and consult a physician.
- Loosen cuffs immediately if pulsation or throbbing occurs as the cuffs may be wrapped too tightly.
- For people with diabetes or vascular disease that require frequent skin assessment; Consult a physician.
- For people who use warming devices in combination with cuffs that require frequent assessment as skin irritation may occur; consult a physician.
- For people positioned in the supine lithotomy position (with or without cuffs) for an extended period and/or require special attention to avoid extremity compartment syndrome; consult; a physician.

QUICK START INSTRUCTIONS

READ THE COMPEX MANUAL THOROUGHLY
(available at compex.com)

- Put the boots on your legs
- Secure them tightly with the hook and loop and zipper
- Hold the On/Off button for 1 second to turn on the device.
- The device will begin to operate in the default function/ mode F1 which will inflate each of the four chambers to the default pressure setting. To change the mode and pressure settings, please read the full User Manual.
- Hold the On/Off button for 1 second to turn off the device.

CHARGING THE CONTROL UNIT

Make certain the charge pin is aligned with the global power plug charging receiver. The battery icon will flash during charging and will hold when fully charged. The device may be used while it is charging.

PRODUCT SUPPORT

Phone: 1-877-266-7398 (877-COMPEX8)
Email: service@compex.com

COPYRIGHT © 2020 by DJO, LLC
Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

DJO, LLC
A DJO Global Company
1430 Decision Street
Vista, CA 92081-8553
U.S.A.

CX2021F02