



READ THE MANUAL BEFORE OPERATING THIS PRODUCT

COMPEX ®

FIXX™ MINI

Massager

INTRODUCTION

Compex® Fixx™ Mini provides the ultimate portable “on the go” deep tissue massage for sore and overworked muscles. Fixx™ Mini adjusts to 3 different speeds ranging from 1500, 2000, and 2500 percussions per minute. Combining speed and hand pressure customizes massage to meet your needs and requirements for recovery. Fixx™ Mini can help reduce soreness, loosen tight muscles, and can be used for warm up prior to activity.

OPERATING INSTRUCTIONS

1. Connect the charging plug into wall outlet. Charge 1-1.5 hours before use.
2. When charging Fixx™ Mini a red light appears. Once the battery is fully charged, a green light appears.
3. Before use, make sure Fixx™ Mini is off. Then, place the ball on to the metal shaft with slight pressure until the shaft stops inside the ball. Turn clockwise until ball is secured in place. The ball must be connected prior to operating the Fixx™ Mini.
5. To turn on, hold power button down until lights come on. Press power button again to start first level speed 1 and press again to increase speed to second level speed 2 and then again to third level speed 3.
6. There is one indicator light on the Fixx™ Mini body showing green for power on. The indicator will show red when battery running lower than 25% of its capacity.
7. To turn off, press and hold power button down.
8. To remove ball, turn Fixx™ Mini off and turn ball counterclockwise and pull ball away from the shaft. Use antibacterial or alcohol wipes to clean ball between uses.

TROUBLESHOOTING

- If Fixx™ Mini will not power on, charge battery until fully charged.
- If ball will not remove from shaft, turn counterclockwise to remove.
- If battery will not charge, check connections of power cord connected to Fixx™ Mini or wall plug in power outlet.
- If Fixx™ Mini disconnects from wand handle, check to assure both tabs are turned and locked into slots of the wand handle end.

MAINTENANCE, CLEANING & STORAGE

- Wipe down with damp towel to clean device, then dry.
- Wipe ball and/or tips and grip with anti-bacterial wipes to prevent any bacteria contamination.

SAFETY WARNINGS

THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS:

- Used by and/or administered by adults only (21 and over).
- Stop immediately if you feel any pain or discomfort.
- Avoid contact with head and/or other bony areas.
- Use on body parts for no more than 1 minute per specific location. Keep moving over muscles and avoid bony areas.
- Avoid putting fingers, hands and/or hair near the ball/shaft section of Fixx™ Mini.
- Do not drop Fixx™ Mini.
- This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- The appliance is only to be used with the power supply unit<model NO. ICP15A-050-2000D> provided with the appliance.

CONTRAINDICATIONS

PLEASE DO NOT USE FIXX™ MINI WITHOUT RECEIVING APPROVAL FROM YOUR PHYSICIAN IF THE FOLLOWING CONDITIONS APPLY:

- Recent surgeries of any kind
- Skin conditions and/or wounds affected by vibrating Minis
- Herniated disks
- Pregnancy
- Diabetes/neuropathy or retinal damage
- Wear of pacemakers
- Injuries affected by vibrating massagers
- Epilepsy or migraines
- Disc herniation
- Spondylolisthesis
- Vertebral detachment or spine joint rigidity
- Recent replacement of joints or IUD
- Metal pins or metal plates
- Any health-related diseases



WARNINGS

1. Do not use if injured.
2. Do not immerse in water, sand and/or loose debris that might get into motor housing vent.
3. Do not remove screws or attempt to disassemble.

ADVERSE REACTIONS

You may experience skin irritation and mild bruising if Fixx™ Mini is used in one spot for more than 60 seconds or over bony areas. Adverse reactions may be experienced by people recovering from surgery, skin conditions, wounds, nerve conditions, broken bones and/or contusions. To avoid reactions, users should consult doctor prior to using Fixx™ Mini.

PERFORMANCE SPECS:

NO LOAD CONDITION (±5%)				LIMITED CONDITION (±5%)			
Speed Level	Speed 1	Speed 2	Speed 3	Speed Level	Speed 1	Speed 2	Speed 3
Percussions per minute (PPM)	1500	2000	2500	PPM	1500	2000	2500
Power(W)	3.45	4.15	4.93	Power(W)	23.68	38.33	54.90
Voltage(V)	11.74	11.49	11.44	Voltage(V)	10.62	10.56	10.32
Current(A)	0.29	0.36	0.43	Current(A)	2.23	3.63	5.32
Noise(dB)	53.7	55.6	58.2	Distance	30cm		
Battery Cell Specification	14500 800mAH 3 series 1 parallel 12.6V						
Adaptor	AC 100V-240V, DC 5V 2A USB charging port, Wire length 1.5M (International plugs)						
Stand-by power consumption	2mA standby mode ≤30uA						
Battery charge & discharge time	Charging time: 1-1.5 hours, Discharge time: 1-1.5 hours						
Working Temp.	-10°C ~ 40°C						
Store Temp.	-10°C ~ 25°C						

BATTERY AND ADAPTOR INFORMATION

- Battery Shelf Life: If the device is not used for an extended period, we recommend that you fully charge Fixx™ Mini every 3 months. You should also store the Fixx™ Mini in a cool and dry environment.
- Service life of battery is 3 years.
- Protective current 10A 3S

DECLARATION OF CONFORMITY:

FCC PART 15B Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

WARRANTY

This device carries a limited warranty of one year from the date of delivery. The warranty applies to the device only, and the accessories are not covered by this warranty.

During the warranty period, defective items will be repaired or replaced at no charge. Any evidence of misuse, abuse, alterations, or externally caused damage may have this warranty invalid.

For more information, please contact the distributor/manufacturer.

CONTACT INFORMATION

Manufactured for DJO, LLC

Vista, CA 92081-8553 USA

Phone: 1-877-266-7398 (877-COMPEX8)

Service@compexusa.com

COMPEX FIXX™ MINI MASSAGER

MODEL: CX202WL06 – (BLK)

Powered by Li-ion battery (10.8V, 850mAh, 9.18Wh)

Rated Input: 5V 2A

Rated Power: Max. 12W

Use only with <ICP15A-050-2000B> supply unit



Made in China

Copyright © 2020 by DJO, LLC
Individual results may vary. Neither DJO, LLC nor any of its subsidiaries
dispense medical advice. The contents of this document do not constitute
medical advice. Rather, please consult your healthcare professional
for information on the courses of treatment, if any, which may be
appropriate for you.

DJO, LLC
A DJO Global Company
1430 Decision Street
Vista, CA 92081-8553
U.S.A. | compex.com

CX1921F13 Rev A