



Instructions For Use OTA136: Analog Holder

The Analog Holder comes with 3 individual chucks.
Red chuck holds 2.0 to 3.0mm analogs,
Gold chuck holds 3.0 to 4.5mm analogs,
Green chuck holds 4.5 to 6.5mm analogs.

To change the chucks, unscrew the bottom piece while holding the chuck down. Replace with desired size chuck. Screw bottom piece back into body while holding chuck in place.

Sterilize prior to use, using standard autoclaving procedures.

Osseous Technologies of America
A Division of A. Titan
10 Centre Drive
Orchard Park NY 14127
www.osseustech.com