



X-OTOMES

by A. Titan

A T R A U M A T I C E X T R A C T I O N S

An Instructional Tip: Beveled tips follow root structure down to PDL. Using a 2-3 degree rotation around circumference of tooth you will gently separate the fibers from the root structure down to the PDL, creating a trough that will minimize tissue and bone trauma.



XO-10
Small straight tip works on anterior teeth



XO-20
Small curved tip works on anterior and single rooted teeth



XO-30
Small angled tips work on the mesial of anteriors & bicuspid



XO-40
Small angled tips work on the distal of anteriors & bicuspid



XO-50
Medium angled tips work on 1st, 2nd, and Premolars



XO-60
Medium angled tips work on 1st, 2nd, and Premolars



XO-80
Straight Micro



XO-110
Notched tip X-Otome



XO-70
Straight spade shaped blade ideal for interproximal tissue reflecting



XO-77
Beveled spade shaped blade slides nicely below gum line and follows the contour of root to both sever fibers and aid in root removal



XO-77D
Angled for the distal, the beveled spade shaped blade slides nicely below gum line and follows the contour of root to both sever fibers and chase ankylosed roots



XO-77M
Angled for the mesial, the beveled spade shaped blade slides nicely below gum line and follows the contour of root to both sever fibers and chase ankylosed roots



XO-77AL
Angled left, the beveled spade shaped blade slides below gum line to sever fibers and chase ankylosed roots



XO-77AR
Angled right, the beveled spade shaped blade slides below gum line to sever fibers and chase ankylosed roots



XO-90
Large angled tip excellent for 3rd molars




XO-100
Large angled tip gives the ability to maneuver and elevate tuberosity around 3rd molar

- Made in the America
- Hybrid instrumentation, a cross between an Elevator and a Periotome
- Custom sure-grip handle for controlled luxation force
- Honed narrow tips to disrupt the periodontal fibers
- Preserves tissue structures
- Eliminates bone fracture and promotes minimally invasive exodontia
- 100% stainless steel

Elemental Kit K-3300

Contains:
XO-10, XO-20, XO-77 and
a sterilization cassette

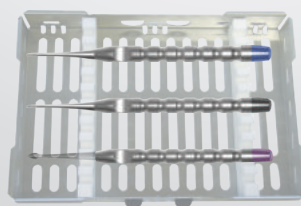

X-Otome Kit K-6600

Includes one of each:
XO-10, XO-20, XO-30, XO-40,
XO-50, XO-60, XO-77 and a
sterilization tray



X-Otome Kit K-7700

Includes one of each:
XO-77, XO-77AL, XO-77AR,
XO-77M, XO-77D and a
sterilization tray

