


Bio Skin® TriLok™ Ankle Control System

Application Instructions

Looking down at your foot

1 Undersleeve

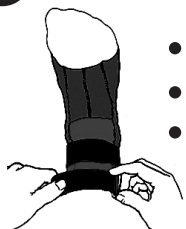

- Pull the sleeve on over foot.
- Secure both 1½" straps with grey tabs.
- Make sure the top strap is tight.


Note: Leave collar with Bio Skin® logo up until step 4.

2 FootLok Strap™




- The white FootLok Strap™ **MUST BE APPLIED** for maximum stability.
- Follow the colored instructions printed on the FootLok Strap™. Start at the narrow end, noting Right or Left foot.

3 Stirrup Strap (Marked with the TriLok™ logo)

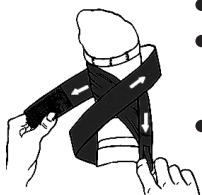

- Open strap by pulling the finger loops on each end.
- With TriLok™ logo facing out, attach strap with logo over achilles tendon.
- Cross each strap over the top of the foot then cross both under the foot. Each end is pulled up snugly and attached to the sides of the ankle.
- Keep strap attached to brace after use.

4 Collar

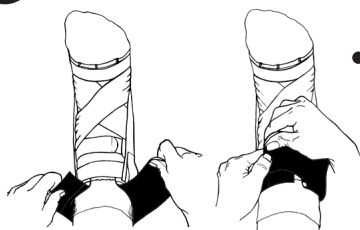

- Roll collar down and secure.



TriLok™

Easy Care Instructions

Wash Bio Skin® regularly to prevent body salt and oil build-up that naturally occurs after the product has been worn a few times. Attach the hook to the loop to prevent lint build-up. This will give your Bio Skin® a longer life span.



Cropper Medical, Inc.
custserv@bioskin.com
800.541.2455
www.bioskin.com

