Instructions for use

Ossur® AFO Light

Life Without Limitations®

.....	3
ⒺⒺ Instructions for use.....	5
ⒺⒺ Gebrauchsanweisung.....	8
ⒺⒺ Guide de fabrication.....	11
ⒺⒺ Instrucciones para el uso.....	14
ⒺⒺ Istruzioni per l'uso.....	17
ⒺⒺ Brugsanvisning.....	20
ⒺⒺ Bruksanvisning.....	22
ⒺⒺ Gebruiksaanwijzing.....	24
ⒺⒺ Instruções de Utilização.....	27
ⒺⒺ 取扱説明書.....	30

EN - Caution: Ossur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Ossur products and components are used with other recommended Ossur or authorized components. If unusual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist.

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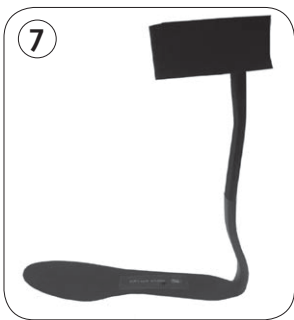
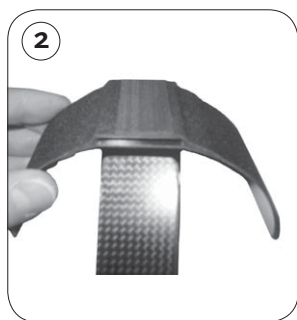
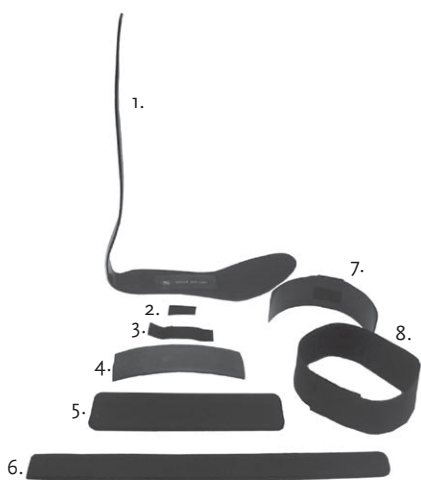
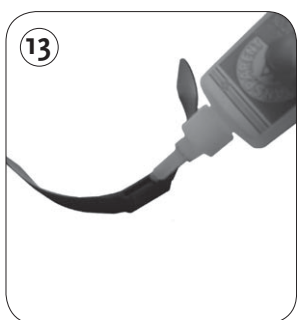
1. FITTING THE AFO TO THE PATIENT

- Apply the J-shape spring to the patients' lower leg (**pic.1**).
- Apply the calf piece (**pic. 2**).
- Verify height of the calf piece (**pic. 3**). To reduce the length of the posterior strut, remove calf piece and grid strut to an appropriate level (approximately at the thickest part of the calf). Do not grind the strut at the heel area as this may jeopardize product performance and strength.

Make sure the strut doesn't reach the back of the knee in seated or slightly kneeled position and does not compress the perineal artery or nerve!

2. ASSEMBLY OF THE AFO

- Slide the calf-piece over the posterior strut.
- Apply slight tension on the elastic band (**pic. 4**) and attach the micro-velcro at that level of the strut.
- Connect the elastic band using the loop-Velcro.
- Apply the tapered part (rounded edge) of the silicon pad to the distal part of the posterior strut and the entire length of the posterior strut covering the elastic band (**pic. 4**).
- Attach the padding to the calf-piece (**pic. 5**).

The calf padding is slightly bigger than the calf-piece to avoid contact between the piece and the leg!

- Insert the AFO into the shoe (section 4) and allow the patient to walk around for a while to verify the fit of the AFO. Decrease or increase padding as needed (section 5).

When the fit is approved, apply the shrink tube to the lower section of the strut. The shrink tube will allow the user to don their shoe easier while protecting the padding:

- Remove the upper part of the padding of the strut
- Remove elastic band from the Velcro on strut
- Remove the calf piece
- Slide the shrink tube all the way down over the posterior strut (**pic. 6**).
- Use the heat-gun to shrink the tube to proper size. Evenly distribute the heat to avoid burning and wrinkles (**pic. 6**). First apply the heat to the inside of the AFO, then the outside.
- Re-apply the calf piece, elastic strap and reapply padding (**pic. 7**).

3. INSERTION INTO THE SHOE (**pic. 8**)

- If present, remove the insert from the shoe.
- Place the AFO inside the shoe ensuring a stable position over the full length of the footplate.
- If necessary adjust the shape of the footplate (by grinding) to the contours of the shoe-sole (Use the insert as a reference for shaping). Do not grind the strut at the heel area as this may jeopardize product performance and strength.
- If present, re-insert the insole on top of the AFO-footplate.

4. DONNING THE AFO

- When donning the product, use the posterior strut of the AFO as a shoe-horn (**pic. 9**).
- Apply stretch onto the elastic part of the strap before attaching it around the leg, this will ensure good fixation of the strap (indicated in **pic. 10**).
- Remove excessive strap material with a pair of scissors.

5. CUSTOMIZING THE AFO

- In case of point pressure at the heel-bone it is advised to apply padding under the existing padding just proximal of the pressure point (Distal part calf) (**pic. 12**). Remove the padding and glue additional padding to the strut before re-attaching the padding.
- For patients with lower activity levels, where the calf piece doesn't move at all it is advised to permanently fixate the calf-piece by application of strong glue (Loctite 495) into the calf-piece before attaching it to the strut (**pic. 13**).
(Riveting of the calf piece to the posterior strut is also allowed.)
The elastic band and the micro-Velcro may be removed in this case.

6. GENERAL REMARKS AND ADDITIONAL ADVICE

1. To shorten the footplate, saw or grind off the appropriate length. Be certain to buff the edges to eliminate sharp edges. To lengthen the footplate, use foot insert of choice, and fixate it to the foot plate.
2. To adjust the biomechanics of the foot, the use of a foot orthotic is advised.
3. Contraindications:
 - Extreme spasticity
 - Severe ankle deformations
4. The AFO cannot be reshaped by heating. Heating will cause delamination and negatively alter the dynamics of the orthosis.
5. The right footwear is critically important for proper function of the AFO. The following criteria should be met when selecting the right footwear to use with this AFO:
 - Firm heel counter – for proper control of the rear foot
 - Laced or Velcro fastening
 - Removable insole – to allow space for the AFO footplate with modifications
 - Closed heel and toe box

AFO LIGHT INDICATIONS:

- Mild to moderate drop-foot secondary to neurological conditions

AFO LIGHT CONTRA-INDICATIONS:

- Excessive ankle instability or excessive deformities
- Severe spasticity
- Extreme activities
- Fluctuation Edema

AFO LIGHT SIZE SELECTION

Product Name	Leg	Size	Foot Size	Length of Posterior Spring	Heel Height
Ossur AFO Light & ACC	Right	Small	8 3/4" - 10 1/4" (22-25cm)	13 1/2" (34cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Left	Small	8 3/4" - 10 1/4" (22-25cm)	13 1/2" (34cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Right	Medium	9 1/2" - 11" (25-27cm)	14 1/4" (36cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Left	Medium	9 1/2" - 11" (25-27cm)	14 1/4" (36cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Right	Large	10 1/4" - 11 3/4" (27-29cm)	15" (38cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Left	Large	10 1/4" - 11 3/4" (27-29cm)	15" (38cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Right	X-Large	11" - 12 1/2" (29-32cm)	15 3/4" (40cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Left	X-Large	11" - 12 1/2" (29-32cm)	15 3/4" (40cm)	9/16" (1.5 cm)

Replacement AFO Light accessory kits sold separately

- Warranty : 6 months
- Impact Levels : All
- Suggested Weight Range: Up to 220 lbs. (100 kg.)

WARNING.

1. The AFO Light is a supportive brace only and is not intended or guaranteed to prevent injury. Ossur accepts no responsibility for any injury which might occur while using this product.
2. With the use of this type of device, it is possible to receive various types of skin irritation and or pressure points. If you experience any unusual discomfort while using this product, consult your health care provider immediately.
3. Patients with foot neuropathy should not use this product.

PART LIST

1. J-shape Spring
2. Velcro dot
3. Elastic band w/ loop velcro
4. Shrink tube (6")
5. Foam pad Calf-piece
6. Foam pad posterior strut
7. Calf-piece
8. Calf-strap

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Nome prodotto	Gamba	Taglia	Misura piede	Lunghezza della molla posteriore	Altezza del tacco
Ossur AFO Light & ACC	Dx	Small	8 3/4" - 10 1/4" (22-25cm)	13 1/2" (34cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Sx	Small	8 3/4" - 10 1/4" (22-25cm)	13 1/2" (34cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Dx	Medium	9 1/2" - 11" (25-27cm)	14 1/4" (36cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Sx	Medium	9 1/2" - 11" (25-27cm)	14 1/4" (36cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Dx	Large	10 1/4" - 11 3/4" (27-29cm)	15" (38cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Sx	Large	10 1/4" - 11 3/4" (27-29cm)	15" (38cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Dx	X-Large	11" - 12 1/2" (29-32cm)	15 3/4" (40cm)	9/16" (1.5 cm)
Ossur AFO Light & ACC	Sx	X-Large	11" - 12 1/2" (29-32cm)	15 3/4" (40cm)	9/16" (1.5 cm)

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Ossur® AFO Light is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.