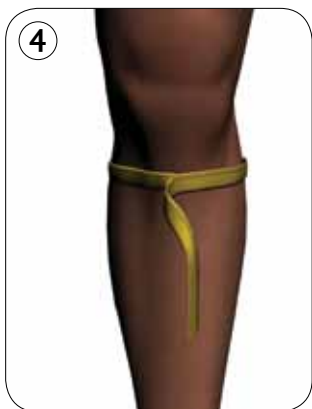
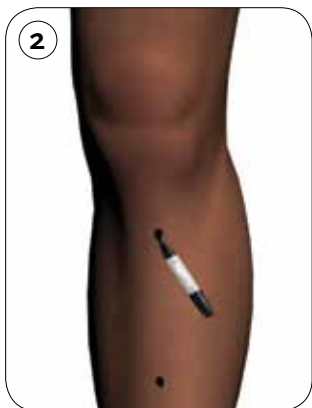
Digital Measuring System for Unloader & CTi CM Braces

ENGLISH

1. Use a marker pen to apply landmarks on the patient's leg (**Figure 1**):
 - The medial joint space
 - Along the the tibial crest at the tibial tuberosity and 15 cm below (**Figure 2**)
2. With the patient standing in a weight bearing position, take the M - L measurement at the joint space (**Figure 3**). Record this measurement on the patient's information label.
3. With the patient standing in a weight bearing position, take from mid patella for the above and below circumference measurements at 7.5cm and 15cm (**Figure 4**). Record these measurements to the patient's information label.
4. Mount the camera to the camera stand. Do not take images unless the camera is mounted to a tripod. With the patient standing in a weight bearing position facing the camera, adjust the height of the tripod until the camera is at mid patella level.

Position the camera so that it is approximately 60 cm – 90 cm away from the patient. Your image must capture the leg 20cm above and below mid patella. Ensure the camera is aligned facing the anterior of the knee to eliminate any internal or external rotation.

Before taking a picture make sure the patient's information label is applied to the patient's thigh, approximately 10 -15 cm above mid patella. Note: It is important it should be captured when the image is taken (**Figure 5**).

5. Turn the patient 90 degrees and move the patient information label to within view of the camera. From the same distance as in Step 4, capture the lateral view (**Figure 6**).
6. Please proceed to the Össur website (www.ossur.co.uk/dms) to complete the online order form and attach the photos.

Please note you must have the following information to proceed:

- Account with Össur UK
- Official account/reference code from your company
- PO number
- All patient information listed at the beginning of the instruction form

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EN - Caution: Össur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Össur products and components are used with other recommended Össur or authorized components. If un-usual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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Össur Americas

27051 Towne Centre Drive
Foothill Ranch, CA 92610
USA

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