

## BA06039 - Allure folding shower chair

### Product Specifications (BA06039)

Length	52cm
Width	52cm
Height	91cm
Seat Depth	40cm
Seat Width	41cm
Seat Height	48cm
Max User Weight	113kg

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact CareCo.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened.
- Take particular care when using the product on damp surfaces.



### Assembly and Maintenance:

To assemble the chair:

1. Open the arm frames so that they are pointing away from the backrest.
2. Lift the seat frame upwards and slot into the plastic connectors half way down the arms frames - you will hear this 'click' into place.
3. Press the seat downwards to connect with the seat frame - this will 'click' into place.

Ensure the chair is secure prior to use. Activate the brakes on the back wheels to lock chair in position. Clean regularly with a standard household cleaner.

## Warranty:

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement.

If you have any queries please contact CareCo.



CareCo, Hubert Road, Brentwood,  
Essex CM14 4JE

[www.careco.co.uk](http://www.careco.co.uk)