



Liteway Tri-Walker

Product Code: WA02020

OWNERS HANDBOOK

www.careco.co.uk

Your CareCo Liteway Tri-Walker



List of Parts

1. Brake Handle/Lever
2. Tightening Handle
3. Quick-Fold Mechanism
4. Rear Wheel
5. Front Castor

Introduction

Thank you for purchasing the CareCo Liteway Tri-Walker with seat. The Liteway Tri-Walker has been designed and manufactured to meet our own high level of specifications and we hope you will be delighted with your purchase.

The Liteway Tri-Walker with seat has been designed as a lightweight and portable Tri-Walker to aid the user to walk and offer stability and security to the user. The Liteway is designed for use both indoor and outdoor and will take a maximum user weight of 115kg (18 stone).

Specification

Tri-Walker Dimensions: 57cm (Length) x 65cm (Width) x 81.5-99cm (Height)

Folded Dimensions: 57cm (Length) x 26.5cm (Width) x 81.5cm (Height)

Weight: 6.5kg

Max User Weight: 115kg


Assembly

The CareCo Liteway Tri-Walker will arrive fully assembled and folded.

Once you have removed all the protective packaging open up the Tri-Walker frame and push downwards on each side of the seat until it clicks into place

Height Adjustment

To adjust the height of the handles, unscrew the tightening knob, adjust handles to desired height and screw in the tightened knob.

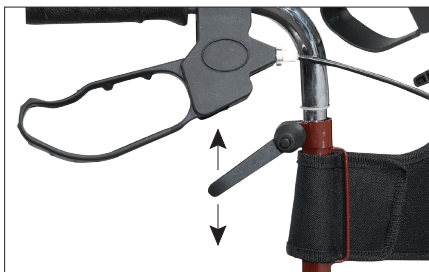

Operation



1. To operate the brakes squeeze the lever up to engage the brake
2. To lock the brakes, push down on the brake lever until it locks in place
3. To unlock the brakes, pull up on the brake lever until the lock is removed

Safety

For Safety:

- Ensure you adopt a comfortable posture and can operate the brakes with ease.
- Ensure the handles are at equal heights and securely locked in position.
- Ensure the tri-walker is fully folded out prior to use.
- Ensure the brakes are tested prior to use.
- Do not use on gradients greater than 7 degrees.
- Ensure the brakes are locked when using the seat.
- When sitting, ensure the user is sat in the centre of the seat.

Folding

For ease of storage and transportation the CareCo Liteway Tri-Walker has an easy fold mechanism.

To fold the Liteway, lift the seat connecting the two sides of the frame up and pull the two sides of the tri-walker together.

Maintenance

Be sure your CareCo Liteway Tri-Walker is in proper working order at all times. Please contact CareCo for any repairs required.

- Ensure the walker is kept clean. Wipe down the walker with a damp cloth on a regular basis.
- Check the wheels and height adjustable handles regularly.
- Ensure the brakes have been adjusted to suit the wheels and user.
- Ensure all nuts, bolts and screws are securely tightened at regular intervals.
- If the solid wheel becomes cracked or severely worn, replace immediately
- Check the front castor wheel swivels freely.

Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the tri-walker or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors and grips, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.



**CareCo, Hubert Road, Brentwood,
Essex CM14 4JE**
www.careco.co.uk

