



# Atlas Rollator

Product Code: WA01055

**OWNERS HANDBOOK**

[www.careco.co.uk](http://www.careco.co.uk)

# Your CareCo Atlas Rollator



## List of Parts

1. Backrest
2. Adjustable parking / running brake
3. Seat
4. Height adjustable handles
5. Swivel front castors
6. Shopping bag

## Introduction

Thank you for purchasing the CareCo Atlas Rollator. The Atlas Rollator has been designed and manufactured to meet our own high level of specifications and we hope you will be delighted with your purchase.

The Atlas Rollator has been designed as a lightweight and portable rollator to aid the user to walk and offer stability and security to the user. Atlas Rollator is designed for use both indoors and outdoors and will take a maximum user weight of 136kg (21 stone).

## Specification

**Rollator Dimensions:** 69cm (Length) x 67cm (Width) x 79-94cm (Handle height)

**Folded Dimensions:** 69cm (Length) x 28.5cm (Width) x 36cm (Height)

**Seat Height:** 53cm

**Weight:** 7.5kg

**Max User Weight:** 136kg

## Intended Use

Designed for indoor and outdoor use on flat surfaces, to assist the user to walk, rest and carry items.

The intended user should possess the necessary physical, visual and cognitive skills to operate the rollator safely. If you have any doubt, please consult your GP.

It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the rollator or surrounding property.

## Safety

This rollator is designed to be used as a mobility aid, creating a natural gait as the user walks securely with their primary weight between the handlebars with the unit in front of the user.

### Do:

- Read this manual carefully before using your rollator.
- Always ensure the brakes are in the locked position if the rollator is left unattended.
- Check all castors and wheels are secure before use.
- Check the brakes are functioning properly.
- Check handle height adjustment and that hand knobs are secure before use.
- Check the rollator is fully unfolded.
- Take care when carrying the rollator when folded. Keep your hands away from the folding mechanism to prevent any risk of pinching.


## Don't:

- Use the rollator as an aid to transport persons, it is not designed as a wheelchair.
- Sit on the rollator if the brakes are not locked, this could cause the rollator to move resulting in injury to the user or other persons.
- 'Drop' into the seat of the rollator. Always sit in the rollator in a smooth, controlled motion. Seek assistance if necessary.
- Hang objects on the brake levers or handgrips. This could affect brake operation and/or stability of the rollator.

Every care has been taken to ensure that your rollator will give you safe, reliable service provided you adhere to the advice given above.

## Assembly

Remove all packaging. The rollator will be folded as shown below. Firstly unfold the front castors and secure in place using the handknobs.

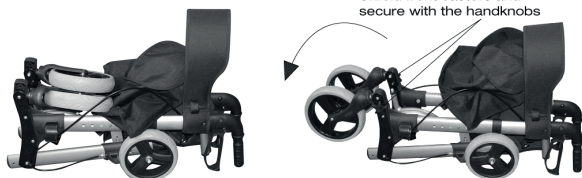

Ensure the thread of the handknob locates through the hole in the castor assembly and that the thread locates in the bolt on the inside of the frame.

Fold out the rear leg and secure the legs in place with the handknobs. Always take care to ensure that the handknob locates through the hole in the leg and into the nut on the frame.



## Opening and Folding

To open your rollator push down on either side of the cross bars as shown below. You should feel both sides 'click' into position.



To fold the rollator gently pull the strap in the middle of the seat as shown below.



## Handle Adjustment

The rollator's handles can be adjusted in height to suit different users. Simply depress the spring button and set to the desired height. Repeat for the opposite side and make sure that both handles are secure and set at the same height before use.



The spring button should be locked in the same position on either side.


## Brake Operation

1. Pull up on brake handle lever to slow the rollator.
2. Push down on brake handle lever to lock the rear wheels, pull up on the brake lever handle to release the lock.

Brakes must be applied in the lock position when the seat is going to be used. Pre-check the lock by pushing forward on the rollator.



Brake adjustment is carried out as follows:

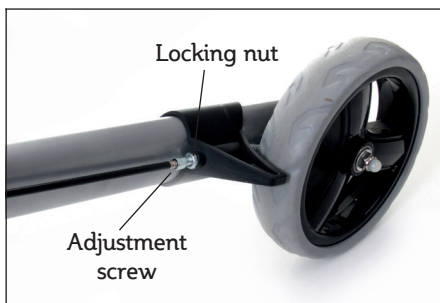

Turn the adjuster screw anti-clockwise to increase the tension then lock the adjustment by turning the locking nut clockwise whilst holding the adjuster screw stationary.

**Brake adjustment reduced to allow light pressure may be ineffective on the brake in the locked position.**

## Maintenance

Over a period of time the brake cables may stretch, and if no further adjustments can be made the cables should be replaced.

The castors should be checked that they are secure each time before using the rollator. Also, check the brakes operate correctly and the handles and hand knobs are secure.

Should any part of the rollator appear loose or insecure discontinue use until it is inspected by an authorised dealer.



## Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the rollator or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors and grips, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.



**CareCo, Hubert Road, Brentwood,  
Essex CM14 4JE**

**[www.careco.co.uk](http://www.careco.co.uk)**

