



Tandem Duo Rollator

Product Code: WA01057

OWNERS HANDBOOK

www.careco.co.uk

Your Tandem Duo Rollator



List of Parts

1. Backrest
2. Rollator bag
3. Folding footrest
4. Front castor
5. Support frame
6. Rollator seat
7. Handbrake

Introduction

Thank you for purchasing the Tandem Duo rollator. The Tandem Duo rollator has been designed and manufactured to meet our own high level of specifications and we hope you will be delighted with your purchase.

The Tandem Duo has been designed as a lightweight and portable rollator to aid the user to walk and offer stability and security to the user. The Tandem Duo is designed for use both indoors and outdoors and will take a maximum user weight of 158kg (24 stone).

Specification

Rollator Dimensions: 96cm (Length) x 62cm (Width) x 86-98cm (Height)

Weight: 10kg

Max User Weight: 158kg

Opening and Folding

To open your rollator push down on either side of the support bars as shown below. You should feel both sides 'click' into position. To fold the rollator gently pull the strap in the middle of the seat.



Wheel Assembly

Fold out the rear leg and secure the legs in place with the handknobs. Always take care to ensure that the handknob locates through the hole in the leg and into the nut on the frame.



Insert rear wheels into frame by depressing the spring button and sliding it into the hole.



Shopping Bag

The shopping bag can be fitted to the front of the rollator and can hold small-sized important objects and documents. Simply attach the basket securely to the support frame.



Handle Adjustment

The rollator's handles can be adjusted in height to suit different users. Simply turn the star screw and set to the desired height. Repeat for the opposite side and make sure that both handles are secure and set at the same height before use.



Brake Operation

1. To operate the brakes squeeze the lever up to engage the brake.



2. To lock the brakes, push down on the brake lever until it locks in place.



3. To unlock the brakes, pull up on the brake lever until the lock is removed.



Rollator to Wheelchair

Rotate the backrest of the rollator as shown in photograph and install the legrests as shown below for maximum comfort.



Installing the legrests

The legrests can swing in / out and are removable for convenience. Pull up on the black plastic handle to unlock the legrest and then swing the legrest in or out, as required.

To remove the legrest, swing the legrest out approximately 45° and pull up on the leg rest.

To reinstall the legrest, insert the leg rest at 45° then swing the legrest inwards until it locks in position.



Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the rollator or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts, such as rubber accessories, castors and grips, which are subject to normal wear and tear and require periodic replacement.

If you have any queries please contact CareCo.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.



CareCo, Hubert Road, Brentwood,
Essex CM14 4JE
www.careco.co.uk



CareCo
the discount mobility experts