

Lavenham & Weymouth Tables - RR07035/RR07036

Introduction

Thank you for choosing an overbed table by CareCo, our design and quality are a guaranteed for comfort, safety and reliability. Our tables are designed and built to meet all your demands for a practical, correct use. This user manual provides some suggestions to correctly use the table you have chosen and offers a lot of valuable advice for your safety. Please read carefully through the manual before using the overbed table. Should you have any queries, please contact your retailer for advice and assistance.

Intended Use

The bed trays are primarily intended for people who have mobility problems, mainly bedridden where it is expected the consumption of meals in bed or chair. However they are adaptable for both hospitals and home.

Note:

Check that the product has not been damaged during shipment. In case of damage do not use it and contact your dealer for further instructions.

- Do not use the product for a purpose not indicated in this manual.
- CareCo declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without previous notice.


General Warnings

Always pay close attention to the presence of moving parts that could cause limb entrapment and injury.

- For proper use of the device refer to this manual.
- For proper use of the device consult your doctor or therapist.
- Keep the packaged product away from any source of heat because the packaging is made of cardboard.
- The shelf-life of the accessory is determined by the wear of parts that cannot be repaired and / or replaced.
- Be careful when children are nearby.
- Do not exceed the maximum payload.

RR07036 - Lavenham Tilt-Top Overbed Table

- A. Big screw – 2pc
- B. Small screw – 2pc
- C. Semi-circles on small table – 1pc
- D. Perforated plate – 1pc
- E. Tools – 3pc
- F. Base – 1pc
- G. Castors – 4pc
- H. Column - 1pc
- I. Big table – 1pc

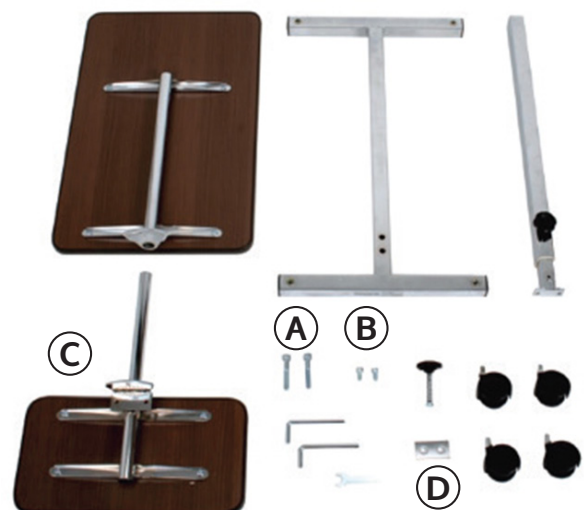

Assembly - Lavenham Tilt-Top Overbed Table - RR07036

Note: Do not remove the safety band before point 3.


1. Unpack the product and check that the content matches with the description of the parts in the manual. Assemble the four castors with the key.
2. Assemble the column using the perforated plate (D), the two big screws (A) and lock them with the big allen key.
3. Assemble the small tray with the small screws (B) using the allen key given.
4. To assemble the bigger tray, insert it in the projecting tube of the small tray. Do not put them closer to each other.
5. Insert the spring of the handles (C) between two semicircles.
6. Put the trays closer to each other, insert the locking screw and fix it with the handle.
7. By turning the handle (fig. 1), the big tray can rotate. Tighten the handle again to lock it at the required inclination.
8. Now the overbed table is ready for use.

fig. 1



RR07035 - Weymouth Tilt-Top Overbed Table

- A. Big screw – 2pc
- B. Small screw – 2pc
- C. Semi-circles on small table – 1pc
- D. Perforated plate – 1pc
- E. Tools – 3pc
- F. Base – 1pc
- G. Castors – 4pc
- H. Column - 1pc

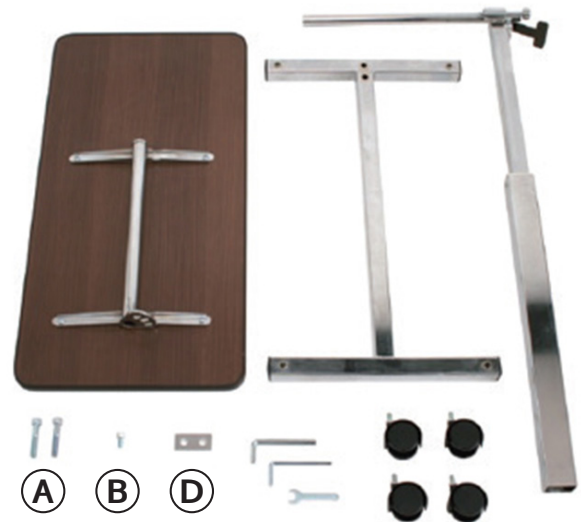

Assembly - Weymouth Tilt-Top Overbed Table - RR07035

Note: Do not remove the safety band before point 3.

1. Unpack the product and check that the content matches with the description of the parts in the manual. Assemble the four castors with the key.
2. Assemble the column using the perforated plate (D), the two big screws (A) and lock them with the big allen key.
3. Insert the tray in the projecting tube of the column.
4. Assemble the tray with the small screw (B) using the allen key given.
5. Now the overbed table is ready for use.

Disassembly

To disassemble do the exact opposite of the assembly instructions.

Before Use

- Always check the status of the mechanical parts to guarantee they are used in total safety.
- Check the device is properly assembled. Pay particular attention to the locking of the knobs.

Precautions For Use

- Do not exceed the maximum load capacity.
- Make sure that the floor surface supporting the overbed table is level.
- Make the adjustment operations without load.
- Check all components are well fixed.
- Always assist the patient during the elevation operation

How To Use


The assistant must place and adjust the overbed table with the maximum care to avoid sudden movements or dangerous oscillations. An assistant has to check any movement of the patient.

RR07036 - Lavenham Tilt-Top Overbed Table

Tighten the handle to lock at the required height. By turning the handle (fig. 2) the big tray can rotate. Tighten the handle again to lock it at the required inclination.

RR07035 - Weymouth Tilt-Top Overbed Table

By pulling the handle (A) the tray will lift up automatically. Pushing on the tray, it will come down. Tighten the lever to lock at the required height. By turning the handle (B), the tray can rotate. Tighten the handle again to lock it at the required inclination.

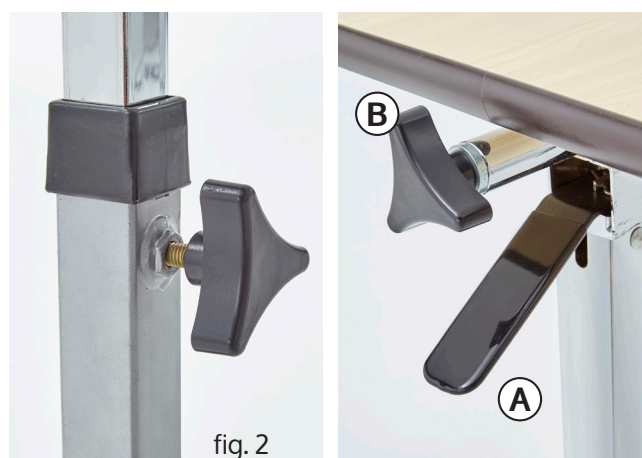

Maintenance

Check the structure periodically: the correct positioning of the tubes, the connections, the hooks, and the complete tightening of screws and / or terminals.

Cleaning and Disinfection

Use a damp cloth or a solution of lukewarm water and neutral soap. Do not use abrasive substances, acids, alkali, alcohol, chlorine-based cleaners, disinfectants and acetone when cleaning, because these substances cause the abrasion of the plastic surfaces and rusting of the metal surfaces.

The manufacture company is not responsible for any damage caused by using materials that can damage the product surface or corrosive chemicals during cleaning.

If you need to disinfect the product, use a common delicate disinfectant neutral detergent.

Conditions of Disposal

When disposing of the device, never use the normal return systems of municipal solid waste. It is recommended instead the disposal through the joint municipal ecological islands for the expected operations of recycling the materials used.

Spare Parts

For spare parts and accessories, please contact CareCo. CareCo shall not be liable for any incident caused by improper use.

Technical Specification

RR07036 - Lavenham Tilt-Top Overbed Table

Height 80-120cm
Small table Width 20cm
Big table Width 60.5cm
Depth 39cm
Weight about 12kg
Pay load 20Kg

RR07035 - Weymouth Tilt-Top Overbed Table

Height 78-110cm
Width 77cm
Depth 39cm
Weight about 12 kg
Pay load 20Kg

- Painted and chromed steel frame
- Manual adjustable height (only for RR07036)
- Automatic adjustable height with lever (only for RR07035)
- Laminate table scratch resistant
- Containment edge made in polypropylene
- Anti-shock and cut resistant edge made in polyethylene
- Reclining table in more positions
- Small table(*only for RR07036)

Warranty

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of receipt. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts. If you have any queries please contact CareCo.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.