




1. Service Plate
2. Leg x 2
3. Shackle with 2 holes
(for left and right side of the wheelchair)
4. Hexagon Screws x 2

How to fit:

1. Use the hexagon screws to attach the shackle to the armrest tube of the wheelchair.
NOTE: The shackle with two holes is for the right side of the wheelchair armrest, the shackle with one hole is for the left side of the wheelchair armrest.
2. Attach the leg to the service place as shown in the image.
3. Secure service plate in place using the shackles.
4. The tray can be fixed onto wheelchair with different seat widths, by pulling the bracket forward or backward until the tray is place in a best position.

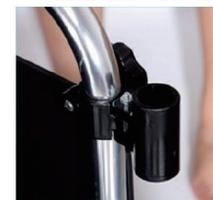

Warranty:

Your CareCo branded product is under warranty to be free from defects in material and workmanship for one year from date of purchase. This warranty expresses our confidence in the workmanship and materials used for our products.

In the event of a defect covered by this warranty, CareCo will, at our discretion, replace the product or supply parts to remedy. This warranty does not cover device failure due to owner misuse or negligence, or due to normal wear and tear. The warranty does not extend to non-durable parts which are subject to normal wear and tear or/and require periodic replacement.

If you have any queries please contact CareCo.



CareCo, Hubert Road, Brentwood,
Essex CM14 4JE

www.careco.co.uk