

Knee Walker

#KN1000



Lifestyle

MOBILITY AIDS

WARNINGS:

DO NOT ATTEMPT TO ADJUST OR OPERATE KNEE WALKER WITHOUT READING ALL INSTRUCTIONS CAREFULLY.

- Before using the Knee Walker, you should be trained by a healthcare professional or dealer.
- Engage the lock and practice bending, reaching and transfers on and off the Knee Walker.
- The use of the Knee Walker is individual to the person. You should develop your own methods for use based on your personal level of function and ability.
- NEVER attempt a maneuver that has not been practiced.
- Be aware of your surroundings when operating the Knee Walker. Look for hazards and avoid them.
- Avoid electrical cords, toys and debris that could be a trip hazard.
- DO NOT use the Knee Walker on or near stairs or escalators.
- Use caution when moving from a carpeted area to hard surfaces or when changing levels, such as from a sidewalk to a street.
- DO NOT use while engaging in the use of mind-altering drugs or while suffering from dizziness.

Assembly Instructions

- Remove contents from carton.
- Insert front tiller into front of frame.
- Once tiller is in place and handles are aligned correctly with the wheels, tighten the screw (using an allen key) by turning clockwise.
- Straighten tiller to the upright position. Slide second screw into receptacle.
- Secure screw and tiller in place by pushing locking handle against the side of the tiller.
- Insert seat post into seat receptacle on frame. Set to desired height and secure in place by inserting thumb screw and turning clockwise until tight.



Operating Instructions

This mobility device provides strength and comfort to anyone who may have an injury below the knee. It allows an even distribution of body weight by supporting half of the weight on the device and the other half on the non-injured leg. The device has a cushion to support both the injured leg and the ankle to make certain that no weight will be placed on the lower leg.

It is designed for easy steering and stopping. Brakes are included on this device for controlled movement and safety.

- Placement of injured leg on bench should be centered side to side and positioned forward to cover the full length of bench.
- With injured leg on bench, stand as straight as possible (adjust the height of the Knee Walker as necessary).
- With injured foot pointing down, the propulsion leg should be maintained as close as possible to the bench.
- While on your toes, start out with small steps with propulsion leg. Concentrate on keeping that leg next to the bench. With practice, you should be able to move forward without deviating to the opposite side.

Handle Adjustments

To adjust the handle height, release the lock tab on the collar and adjust the handle to the desired height. Push the lock tab back to tighten.

Seat Adjustments

To adjust the seat height, flip the lever located under the seat, locate the round circle ring near the frame. This is spring loaded and can be pulled to adjust to desired height and flip lever back to secure seat post.

Hand Brake Use

- To use the brake, simply pull the lever with your finger tips toward the handle bar.
- To lock the brake, pull the brake lever towards the handlebar and push down spring loaded push button (located on top of brake). When done correctly, push button will remain down and brake lever will be locked in place. Repeat for other brake.
- To release brake, simply pull lever towards the handle bar. Push pin will pop up automatically.

Lifetime Limited Warranty

Your Lifestyle Mobility Aids product is warranted to be free of defects in workmanship and materials for the lifetime of the original consumer.

This device was built to exacting standards and was carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will at our option, repair or replace the device.

This warranty does not cover device failure due to misuse or negligence, or normal wear and tear. This warranty does not extend to non-durable components, such as rubber accessories, castors or grips, which are subject to normal wear and tear and need periodic replacement.

If you have any questions about your Lifestyle Mobility Aids device, please contact your authorized Lifestyle Mobility Aids dealer.

©1999 Mini Traveler Co., Inc. All rights reserved.

Lifestyle Mobility Aids is a trademark of Mini Traveler Co., Inc.
Seminole, FL 33775 USA