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The Alecto BDM-100 is a blood pressure monitor that has been developed for easy and accurate blood pressure measurements at the wrist.

Blood pressure monitors should always be positioned at an appropriate measuring position at heart level for reliable measurements.

Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. That is why readings can fluctuate when taken at different times of the day.

Regular blood pressure measurements taken at home can provide your doctor with valuable information on your actual «everyday» values.

Standard values:

Blood pressure (mm Hg)	Normal hypertension	Mild hypertension	Severe hypertension
upper value (SYS)	Up to 140	140-180	Over 180
lower value (DIA)	Up to 90	90-110	Over 110

Important:

- This product is intended for household use only.
- It is NOT usable to measure the bloodpressure at children.
- Keep product and batteries away from children.
- People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves, since deviations in blood pressure values may occur in such cases.
- If you are under medical treatment or receiving medication, please consult your doctor in case of any doubt.
- The use of this blood pressure monitor is not intended as a substitute for consultation with your doctor.

CE

This product conforms to the provisions of the EC directive 93/42/EEC (Medical Device Directive).

The declaration of conformity is available on the website
WWW.ALECTO.INFO

PREPARATION

Batteries:

1. Remove the battery cover and insert two batteries with correct polarity (see the symbol in the battery compartment).
2. Replace the battery cover.

Only discard of empty batteries. They should not be disposed off in the household waste, but at appropriate collection sites or at your dealer.

Time and date:

Because the time and date of a measurement are stored together with the measuring-results, the actual time and date must be set:

1. press button **POWER** to switch-ON the BDM-100
2. press and hold button **TIME** until after appr. 5 seconds the year indication starts to flash, you may now release this button
3. use buttons ▼ and ▲ to set the correct year and press button **TIME**, the month indication starts to flash
4. repeat instruction 3 for the date, hour and minute setting

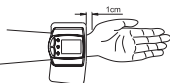
MEASURING

Key rules for accurate measurement:

- Always take readings at the same time of day, ideally in the morning, under the same conditions.
- Do not measure within 30 minutes after smoking, coffee or tea consumption.
- While taking a measurement, sit down, relax, keep still, do not move or speak.
- Hold the appliance at heart level.
- Wait approximately 3 minutes before repeating a measurement.
- For every blood pressure measurement at the wrist an appropriate measuring position at heart level is essential for accurate results.

Equipment setup:

1. Expose the left wrist.
2. With the lower arm free of clothing and jewelry, attach the cuff to the bare wrist. Insure that the cult is positioned about 1 cm above the palm.
3. Fold the outer section back to snugly attach and fasten to the Velcrotape.

Note: Blood circulation to the hand should not be impaired or constricted. Clothing, elastic bands or other constrictive objects must be removed.

Measurement:

- In a sitting position and with the cuff attached as previously mentioned, a blood pressure reading can be obtained.
 - For accurate pressure measurement, take bloodpressure readings in a sitting position.
 - Ensure that the lower arm is raised so that the upper edge of the cuff is on the same level as the heart. You may use the storagebox as a support for your wrist.
 - Ideally, place the palm of the hand facing upwards.
 - In order to ensure a proper measurement, sit quietly and avoid talking. Rapid movements or other activities may alter blood pressure readings.
 - At any time, to stop the pressurizing process, press the **POWER** button; the cuff will stop inflating and air will exhaust from the unit.
-
1. Press button **POWER** to switch-ON the BDM-100 and wait until the display stops flashing.
 2. Use button **PAGE** the select whether the measuring results will be stored in memory page-1 or in memory page-2.

3. Press **START** to activate the measurements
 - the cuff will inflate automatically (a 'P' on the display)
 - when the required pressure is reached, the cuff will deflate and the bloodpressure is measured
 - after detecting the systolic, the diastolic and the pulse, the measured values will be displayed on the LCD and will be stored in the selected memory (page-1 or page-2)
4. Press button **POWER** to switch-OFF the BDM-100

Memory:

The measuring results are automatically stored in memory, including the time and date of the concerning measurement.

1. Use button **PAGE** to select memory-1 or memory-2
2. By repeatedly pressing button **MEMO**, the results of previous measurements will come on the display.

The memory-capacity is 60 measurements, 30 for page-1 and 30 for page-2.

STORAGE AND CLEANING

Clean the housing of the product with a damp, soft cloth. Stains on the cuff can be carefully removed with a damp cloth. Do not wash or dry clean the cuff. Do not store the product in a place exposed to direct sunlight or high humidity.

SPECIFICATIONS

Model:	BDM-100
Operating principle:	Oscillometric method
Indicator:	Liquid Crystal Display
Measuring range:	20 – 280 mm Hg (blood pressure) 40 – 195 / min (pulse rate)
Accuracy:	± 3 mm Hg (blood pressure) ± 5% (pulse rate)
Inflation:	Automatic
Power supply:	2 batteries, type LR 03 (AAA)
Operating temperature:	+ 10 °C – + 40 °C
Storage temperature:	– 20 °C – + 60 °C
Humidity:	30% - 85%
Cuff:	Suitable for wrist circumference 13.5 to 20 cm

TROUBLESHOOTING:

Nothing displayed when you push the Start button:

- Are the batteries exhausted? Replace the batteries with new ones
- Is the polarity of the inserted batteries wrong? Insert the batteries in the correct position

“Err P” is shown on LCD panel:

- Fail to inflate. Please check if the cuff is too loose and fasten it.

“Err 1” is shown on LCD panel:

- Deflate too fast. Please send it to service point

“Err 2” is shown on LCD panel:

- Movements occur on wrist. 1.Keep still after you push the Start button. 2.Other interference signal around BDM-100. Please remove interfering sources such as Magnetic material or cell phone.

“Err 5” is shown on LCD panel:

- Incorrect measured result. 1.The cuff might place incorrect. Please re-wrap the cuff in correct position. 2.You might use it incorrectly, such as measure pressure value on child.

If correct measurement is impossible even after checking the above-mentioned points, consult at the store where you have purchased the unit or the nearest dealer without touching the internal mechanism.

In some cases, there may be error due to the physical condition of the person. In such cases, please consult your doctor.

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- Desinfla demasiado deprisa. Llévalo a un S.A.T. autorizado.