


# MEENTA

# POOLING

1. Access Meenta's Pooling Portal with your Meenta credentials (created at checkout): <https://pooling.meenta.io/welcome>

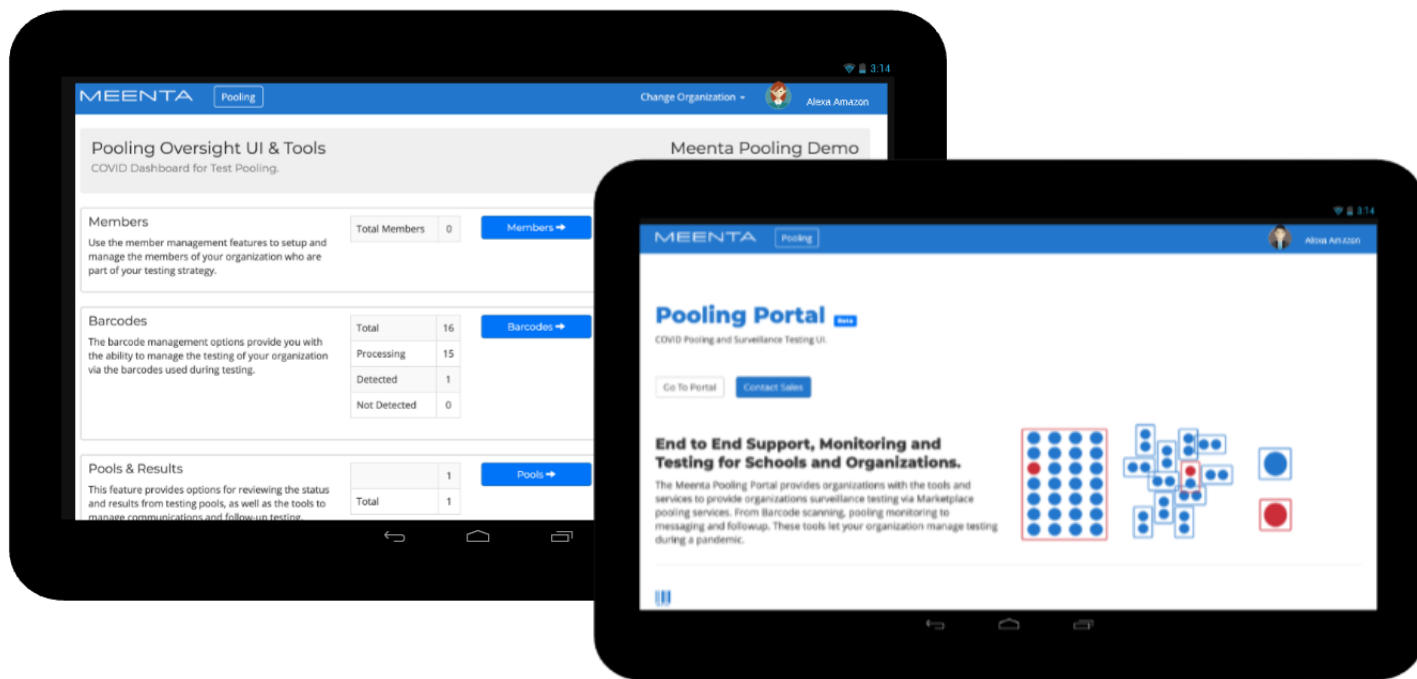


2. Begin by Uploading your Member's Roster of people who you want to get Tested.

3. Receive Your Test Kits and start scanning barcode id's to attach test tubes to individuals in the Pool.



4. Wait 24 hours for your Pool results and push notifications to your Members.

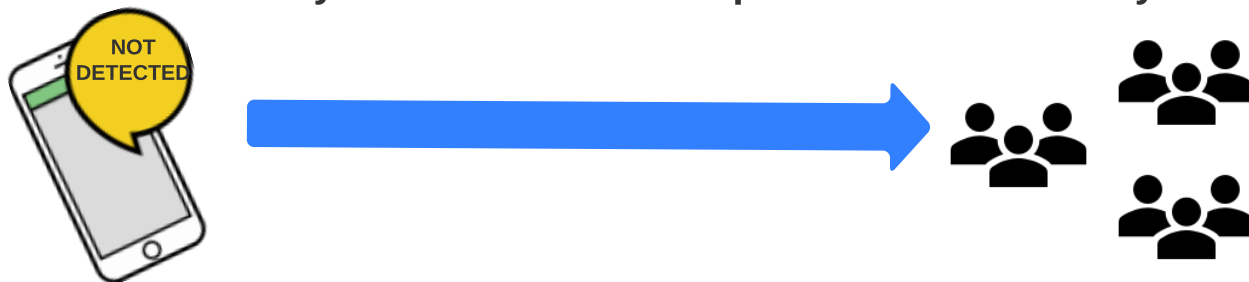

5. Reflex your final pool of Two to Individual Diagnostic Tests through the Meenta Testing Platform

# MEENTA

# COVID TESTING

