

Mercury 6

HANDHELD MAGNIFIER & READER USER GUIDE

trysight

Version 1.2

Congratulations on the purchase of your Mercury 6 Handheld Magnifier with Speech. Please read the manual fully before using. Mercury 6 is an advanced handheld magnifier with accurate reading capabilities and voice control.

Enclosed Parts:

Power Charger

Charging Cable

User Manual

Mercury 6 Handheld Device
with attached legs

DEVICE LAYOUT




IMPORTANT: DO NOT HOLD DOWN THE POWER BUTTON AND VOLUME BUTTON AT THE SAME TIME.

POWER ON/OFF

Hold down the Power button for 3 seconds to start the unit. The unit can take upto 1 minute to start. During this time you will see a boot logo. The unit will start in live magnification mode. If the unit doesn't start, the likely cause is the battery is not charged. To charge the battery, plug in the power adapter to an electrical outlet

and use the provided USB cable to connect the device to the charger as shown below.

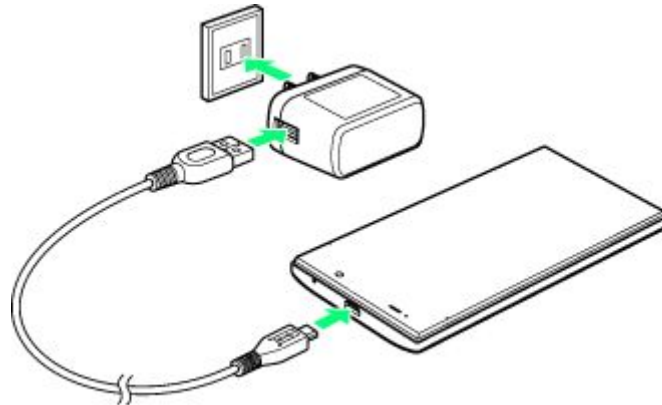

After charging for 30 minutes the unit should have sufficient power to start. To power off the unit, hold down the power button for 3 seconds.

QUICK ON/OFF

To enable quick on/off functionality, the unit can go into "Sleep" mode. The unit can wake up instantly from sleep mode in 1 second.

However, Sleep mode uses a small amount of battery as the unit is not completely turned off. After a few days in sleep mode, the battery will drain out and will need recharging even if you don't use the unit.

To put the unit to sleep, tap

the Power button quickly (Don't hold it down). You will hear a beep and the screen will turn off. To wake it up, tap the power button again.

INTEGRATED LEGS

Mercury 6 incorporates integrated foldable legs. The legs allow for easy positioning of the device over a document. The legs are comprised of a small leg and a

big leg. Open the big leg first, then the small leg. To close the legs, repeat the process in reverse.

MAGNIFICATION MODE

The functions of Mercury 6 can be controlled using simple touch gestures. The left 20% of the screen is designed to ignore any touches to prevent accidental triggers while holding the device.

1) Single Finger Tap to Focus




*2) Two Finger Pinch to Zoom
In/Out or say "Make Bigger" /
"Make Smaller"*



3) Three finger tap to change color or say “Change Color”. To lock the currently selected color mode hold with 3 fingers for 5 seconds until you hear a beep. To unlock the color mode, repeat the same action. When a color mode is locked, changing the color will only cycle between original and the

chosen color mode instead of cycling through all modes.

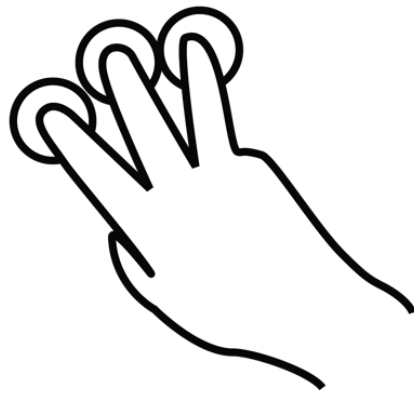

4) Four Finger Tap to Toggle LED Flash Light



5) Five Finger Tap to Open the Menu



6) To freeze/unfreeze the image, tap the pause/resume button on the bottom right.

READING MODE

Position the device over any document. Note the device only reads what is visible on the screen, so it is recommended to zoom out all the way to capture larger documents. Tap on the red circle on the top right or say “Take Picture” to begin reading what is on the screen. Ensure you hold the device steady until it finishes saying “Picture Taken” to capture a

clear image. After you have taken the image, the device will need a few seconds to process the document. A ticking sound will indicate that the recognition is in progress. Reading should start automatically upon completion of this step.

1) Pause/Play

By default, the unit will start reading upon completion of

processing. To pause, tap the pause button on the right side. To resume, tap it again.

2) Zoom In/Out

You can zoom in/out in reading mode using the same 2 finger pinch gesture as in Magnification Mode. Note voice commands don't work in Reading mode.

3) Change Color

To change color, tap with 3 fingers on the screen.

4) Forward/Rewind

Skip Sentences by tapping the forward/rewind arrow while the unit is reading. To read words individually, pause the reading using the pause button and then tap the

forward/rewind arrows to hear individual words.

5) Exit Reading Mode

Press the X on the top right to exit reading mode.

LOW BATTERY WARNING

When the unit has approximately 15 minutes of usable charge remaining it will

alert you by beeping. The beep is a sequence of 4 quick beeps. You will hear this beep every few minutes until the device powers off by itself.

trysight