

Sleep8 CPAP cleaner Manual

[Sleep8 CPAP Cleaner](#) is an easy-to-use device. You can follow the 3 easy steps mentioned below to use the device effectively:

Wipe CPAP accessories. Put them in the bag. Zip it closed. Ensure your mask has been wiped clean to remove any visible dirt or oils, prior to use. Unzip the bag and find the round white valve inside. Detach the tubing from your CPAP machine and attach it to this valve. Put all CPAP tubing, mask, cushions, and dry water chamber inside the bag. Then zip the bag closed.

Connect Sleep8 outside the bag. Slide your Sleep8's triangular port onto the triangular valve on the outside of the bag. Make sure to match the blue dots on the device and the bag's port. They should fit snugly together. The device will only work once there's a secure connection. Again, make sure the bag is completely zipped closed.

Press start & relax. Press the power button until you hear a beep and see the [Sleep8](#) screen light up. The screen will count down until the sanitization is complete. It's normal to hear the fan oscillate and pause for short periods. When done, your Sleep8 will automatically turn off. The bag should be left closed for 1 hour after the cleaning is completed. Afterward, unzip the bag to find your sanitized CPAP accessories ready to use.



[Sleep8 CPAP cleaner](#) is a light-weight and compact portable CPAP cleaner that uses activated-oxygen (ozone) to sanitize your CPAP equipment. The ozone produced by the [Sleep8](#) is utilized to efficiently kill 99% of pathogens consisted of within your CPAP mask, tubing, and humidifier chamber. [Sleep8 CPAP sanitizer](#) is widely compatible with all CPAP and BiPAP devices, masks, and humidifier.