

When shipping specimens via **FedEx** please follow these guidelines:

Place specimen in biohazard bag along with a copy of the requisition form.

Then place all specimens in a large blue biohazard bag along with one absorbent pad (the small white tissue type pad that we have provided for you).

Place the large blue biohazard bag containing all the specimens into the shipping box. Include a copy of the log sheet if available.

Securely tape the shipping box.

Complete the FedEx USA Airbill.

1. Complete all of section #1
2. Section #4A , Express Package Service, please mark FedEx Standard Overnight
3. Section #5 Packaging, mark other.
4. Section #7 Payment, mark recipient.

Please keep the top copy (Sender's Copy).

Place FedEx Airbill on top of shipping box.

Call FedEx toll free 1-800-463-3339 for package pick up.