


Bayonet Connector Change Notification

Dear Customer,

As of August 31, 2017 Hokanson no longer sells cuffs and instruments with Luer connectors. This decision was based on guidance issued by the United States FDA intended to mitigate the risks of misconnections with small-bore connectors, for patient safety.

Hokanson highly recommends that your existing Hokanson cuffs and related equipment be permanently converted to the new Bayonet connectors to avoid future compatibility issues.

For more information please contact Hokanson at 425-882-1689 or toll free at 800-999-8251 or via email at info@deh-inc.com. Or visit the Hokanson website at: www.hokansonvascular.com

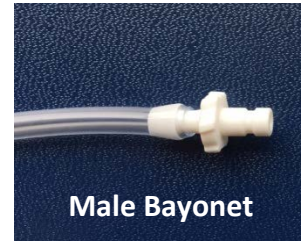

Male Bayonet



Female Bayonet

Cuff Conversion: Female Luer to Male Bayonet

Price: \$35.00

Kit includes:

- (10) male Bayonet
- (1) female Bayonet with 5/32" barb
- (1) female Bayonet with 1/8" barb

1. Cut off the female Luer connector.
2. Use alcohol to dampen the male Bayonet. This will ease the installation process.
3. Insert male bayonet.
4. Convert the inflator to Female Bayonet using the same process.



Female Luer



Male Bayonet




Female Bayonet

Bayonet Adapter Kit:

Price: \$25.00

Kit Includes:

- Male Bayonet to Male Luer
- Female Bayonet to Female Luer
- Connectors connected via 3" tubing

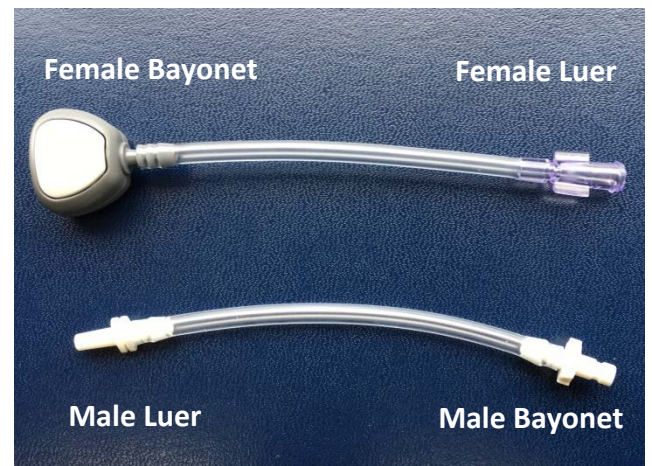

Cuff Conversion Jumbo Pack.

For converting systems, cuffs and accessories.

Price: \$120.00

Kit includes:

- (30) male Bayonet
- (10) female Bayonet with 1/8" barb

1. Cut off the old connector.
2. If needed, use a small needle nose pleyer to stretch the tubing to accommodate the connector barb.
3. Use alcohol to dampen the Bayonet. This will ease the installation process.
4. Insert the bayonet connector.

