

# IMPORTANT

## Product Description

HUBBLE® Daily lens (Methafilcon A) is visibility tinted, with UV block, supplied in a sterile state, immersed in a buffered saline solution in a sealed container.

## Lens Specification

Basic Curve: 8.6

Diameter: 14.2

Center Thickness: 0.04 ~0.30 mm

Powers: +.50 through +4.00 (in .25 steps); +4.00 through +6.00 (in .50 steps); -.50 through -6.00 (in .25 steps) and -6.00 through -12.00 (in .50 steps).

## Effects

Vision correction.

## Indications (Uses)

HUBBLE® Daily lenses are intended to use for daily wear for the correction of refractive ametropia (myopia, hyperopia) in aphakic or not-aphakic persons with non-diseased eyes who may have 2.00 D of astigmatism or less.

## Contraindications (Reasons not to use)

DO NOT USE the lenses when any of the following conditions exist: \* Acute and subacute inflammation or infection of the anterior chamber of the eye. \* Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids. \* Severe insufficiency of lacrimal secretion (dry eyes). \* Corneal hypoesthesia (reduced corneal sensitivity), if not-aphakic. \* Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses. \* Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses. \* Any active corneal infection (bacterial, fungal, or viral). \* If eye becomes red or irritated.

## Precautions

1. Visit eye doctor or practitioner and approach training before initial wearing to ensure your ability to wear/remove the contact lenses. 2. Your eyecare practitioner will recommend a wearing regime, but if in doubt do ask for advice. Always follow recommendations and make regular visits. 3. Daily disposable lens is for single use only. Do not reuse the daily disposable lens because it may cause eyes injury. 4. Daily disposable lens is intended for daily wear not for extended wear. 5. Inform your health care doctor and practitioner that you are a contact lens wearer.

## Side Effects

The patient should be informed that the following problems may occur: \* Eyes stinging, burning, itching (irritation), or other eye pain. \* Comfort is less than when lens was first placed on eye. \* Feeling that something is in eye such as foreign body or scratched area. \* Excessive watering (tearing) of the eyes. \* Unusual eye secretions. \* Redness of the eyes. \* Reduced sharpness of vision (poor visual acuity). \* Blurred vision, rainbows, or halos around objects. \* Sensitivity to light (photophobia) \* Dry eyes.

## Instruction for Use

Always wash and dry your hands before handling your lenses. Initially and on replacement remove lens from packaging and check that the seal of the packaging is intact and the lens is immersed in fluid. Do not use if the packaging or the lens is not intact.

## Lens Insertion and Removal

**Right Lens Insertion:** 1. Ensure that the lens is not inside out. You can verify the shape when you place the lens on the tip of your right index finger as you prepare to insert. (fig. 1 and 2). 2. Hold up the upper eyelid using left hand's index and middle fingers while holding down the lower eyelid with your right hand's middle finger. You can now place the lens onto the eye. (fig.3) 3. Blink a few times in case you do not see clearly immediately. 4. Repeat the procedure for the left eye.

**Removal:** 1. Wash and dry your hands. 2. Look up and while holding your right lower lid down, gently pinch the bottom edge of the lens and lift out of your eye with the pads of your index finger and thumb. 3. Repeat the procedure for the left eye. (fig. 4)



## Emergencies

If the patient has been splashed with chemicals of any kind into the eyes, the patient should: FLUSH EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT THE EYECARE PRACTITIONER OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.

## Recommended Initial Wearing Schedule

Although practitioners may have developed their own initial wearing schedules, the following sequence is recommended as a guideline. Please be cautioned to carefully follow the wearing schedule recommended by the eyecare practitioner regardless of how the lenses feel.

Maximum wearing Time:

Day	1	2	3	4	5	6	7	8	9	10 and after
Wearing Time (Hours)	4	5	6	7	8	9	10	11	12	All waking hours

## Do's and Don'ts

\* Do remember that the health of your eyes is essential for good sight. Don't put it at risk through negligence or vanity. \* Do remove your lenses and consult your eyecare practitioner if any eye reaction occurs such as pain, redness, burning sensation, excessive watering, increasingly blurred vision, colored haloes around light sources and/or sensitivity to light. \* Do consult your eyecare practitioner if you find difficulty to remove the lens from your eye \* Do remove your lenses before going to sleep (unless otherwise advised by your eyecare practitioner). \* Do attend your routine follow up appointments. These are essential to monitor the health of your eyes. Symptom-free wear does not necessarily mean that your lenses are not causing undesirable side effects. \* Do comply with your eyecare practitioner's guide to wearing schedules. Over wear can have serious consequences. \* Do keep your nails short. Long nails harbor bacteria and a scratched eye is vulnerable to infection. \* Do replace lenses if they become damaged or scratched, or if they have deposits which cannot be removed. \* Do wear makeup if you wish, but insert your lenses first. Remove your lenses before you take off your make up. Fiber free mascara and oil free types are best. \* Do remove your contact lenses before you swim, shower, or enter a hot tub. \*Do contact your eye care professional before using any medicine. \* Don't expose your contact lenses to any water: tap bottled, distilled, lake

or ocean water. \* Don't share your contact lenses with others. \* Don't ever use household products (e.g. disinfectants) with your lenses. They can seriously damage your eyes. \* Don't use saliva to wet your lenses. It is teeming with potentially harmful bacteria. It is not true that your own bacteria are safe. \* Don't allow hair lacquer or other cosmetic aerosols to come into contact with your lenses. \* Don't wear lenses in the presence of noxious or irritating fumes.

The following symbols may appear on the labeling or on the package:

SYMBOLS					
 / EXP	Use-by date / Expiry		Batch code		Sterilized using steam or dry heat
BC / B.C.	Base Curve	LS / DIA	Diameter	PWR / SPH / D / DP / P	Diopter / Spherical Power
	Manufacturer		Caution: consult accompanying documents		
	CE marking for conformity		Authorized representative in the European Community		
	Do not re-use		Recycling		
	Prescription only / Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed practitioner				

**YOUR EYES ARE PRECIOUS**

Please read about your new contact lenses' Instruction for use

 **ST. SHINE OPTICAL CO., LTD.**  
4, 5F., No. 276-2, Sec. 1, Ta Tung Rd.,  
Hsi Chih Dist., New Taipei City 221, Taiwan

Version#

  0120 

 **EMERGO EUROPE**  
Prinsessegracht 20, 2514 AP The Hague,  
The Netherlands