



Xpert® Xpress Flu and Xpert® Xpress Flu/RSV

Fast Accurate Diagnosis When it Matters Most.



The Cepheid Xpert **Xpress** Flu and Xpert **Xpress** Flu/RSV tests are qualitative, automated molecular, real-time PCR tests providing accurate detection of and differentiation of influenza A, influenza B, and RSV viral RNA.

Introducing Xpert **Xpress** Flu and Xpert **Xpress** Flu/RSV

- CLIA Waived molecular RT-PCR testing
- Simple 3-step test procedure
- Test time 20 minutes*
- Multiple sample types: Nasal Swab or Nasopharyngeal Swab
- Internal QC in each test cartridge
- Reagents stored at room temperature
- No additional steps once testing has begun
- More than one patient specimen can be run at a time^
- Accurate results with NO confirmation required

* With early assay termination (EAT) for positive Flu or for positive RSV only. Reporting negatives and combined Flu and RSV results in 30 minutes.

^ More than one patient specimen can be run at a time on instruments with more than one module.

Xpert **Xpress** Flu/RSV and Xpert **Xpress** Flu are *in vitro* diagnostic devices FDA cleared for use as CLIA waived or moderate complexity tests.

For *In Vitro* Diagnostic Use.



“Molecular testing (RT-PCR) is recommended as the front line test by the Infectious Diseases Society of America.”

Fleming-Dutra KE, et al. Prevalence of Inappropriate Antibiotic Prescriptions Among US Ambulatory Care Visits, 2010-2011. JAMA. 2016 May 3;315(17):1864-73.

As simple as 1, 2, 3 • Hands-On Time <1 minute

1

Obtain swab specimen



2

Transfer sample to cartridge



3

Insert cartridge and start test



Xpert® Xpress Flu and Xpert® Xpress Flu/RSV

CATALOG NUMBER

Xpert Xpress Flu (10 tests) XPRSFLU-10

Xpert Xpress Flu/RSV (10 tests) XPRSFLU/RSV-10

Specimen kit required for primary sample collection:

Xpress Nasopharyngeal Specimen Collection Kits (100 Nasopharyngeal Swabs & UTM kits) SWAB/B-100

Xpert Nasal Specimen Collection Kits (100 Nasal Swabs & UTM kits) SWAB/F-100

Technology	Real-time RT-PCR					
Sample Type	Nasal Swab, Nasopharyngeal Swab					
TAT	As early as 20 mins for positives* and about 30 minutes for negatives					
Xpert Xpress Flu/RSV	Moderate			Waived		
Nasal Swab [^]	Flu A	Flu B	RSV	Flu A	Flu B	RSV
Positive Percent Agreement	98.9%	98.4%	98.2%	98.9%	98.4%	98.2%
Negative Percent Agreement	97.5%	99.3%	99.1%	97.5%	99.3%	99.1%
	Moderate			Waived		
Nasopharyngeal Swab [^]	Flu A	Flu B	RSV	Flu A	Flu B	RSV
Positive Percent Agreement	100.0%	100.0%	97.9%	97.6%	97.3%	98.2%
Negative Percent Agreement	98.0%	99.0%	98.7%	98.2%	99.6%	98.5%
Sample Storage	15-30 °C for up to 24 hours or 2-8 °C for up to 7 days in transport medium					
Kit Storage	2-28 °C					

* With early assay termination (EAT) for positive results. Reporting negatives and combined reporting in 30 minutes.

[^] Refer to the most current Xpert Xpress Flu/RSV Package Insert 301-7239 (CLIA Complexity: Moderate) and 301-8205 (CLIA Complexity: Waived) for complete details. Refer to the most current Xpert Xpress Flu Package Insert 301-7268 (CLIA Complexity: Moderate) and 301-7320 (CLIA Complexity: Waived) for complete details.

Xpert Xpress Flu/RSV and Xpert Xpress Flu are *in vitro* diagnostic devices FDA cleared for use as CLIA waived or moderate complexity tests.

CORPORATE HEADQUARTERS

904 Caribbean Drive
Sunnyvale, CA 94089 USA

TOLL FREE 1.888.336.2743
PHONE 1.408.541.4191
FAX 1.408.541.4192

EUROPEAN HEADQUARTERS

Vira Soleih
81470 Maurens-Scopont France

PHONE 33.563.82.53.00
FAX 33.563.82.53.01

www.Cepheid.com