


CARR NARCOTICS LOCKER  
MODEL NL20

DATE: 3/28/13    SCALE: NOT TO SCALE  
DRAWN BY: *ggR*

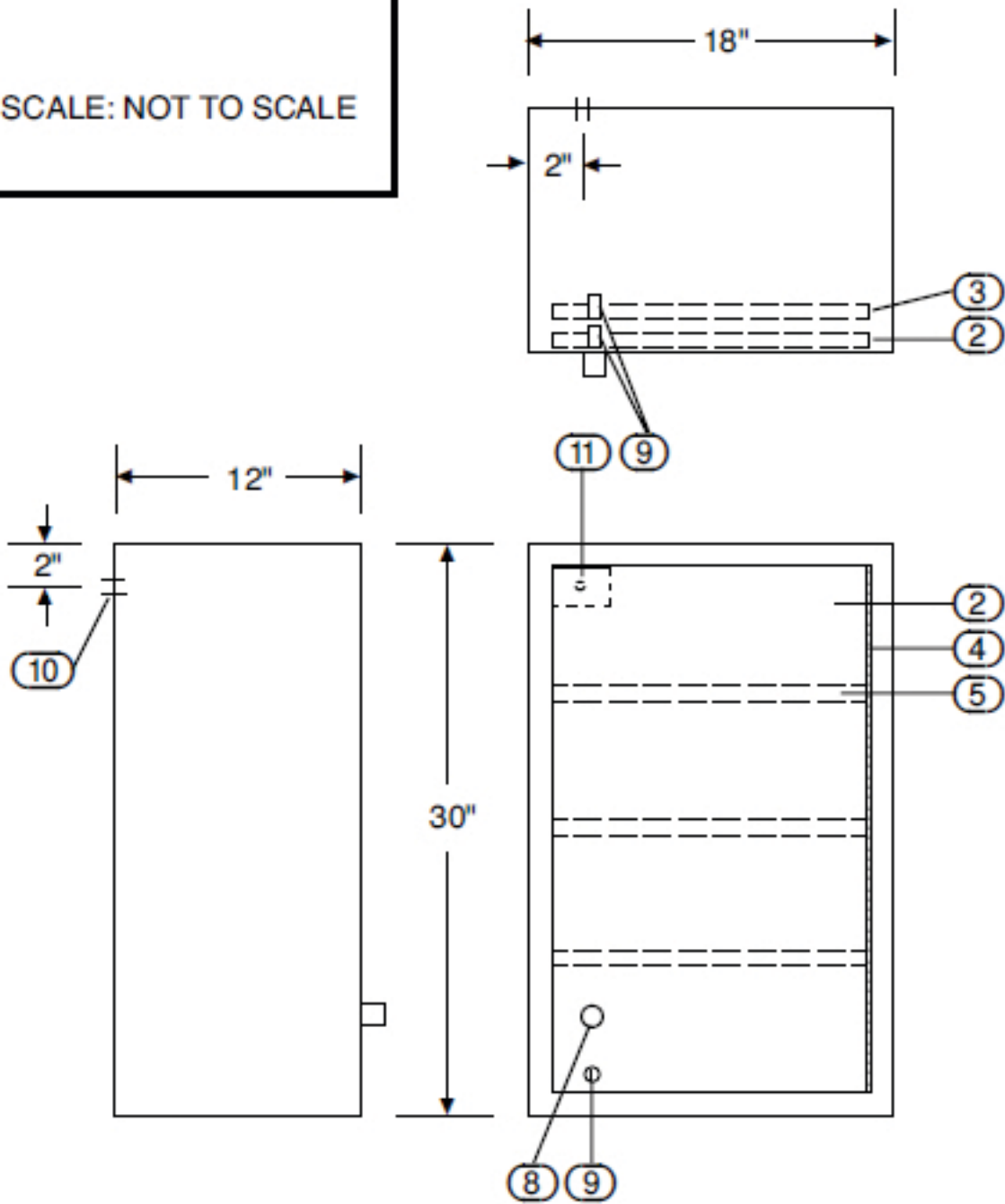


- 1. TYPE 304, #4 FINISH STAINLESS STEEL, DOUBLE WALL CONSTRUCTION THROUGHOUT.
- 2. DOOR, OUTER, DOUBLE WALL.
- 3. DOOR, INNER, DOUBLE WALL.
- 4. PIANO HINGE, FULL HEIGHT, SS.
- 5. SHELF, HEIGHT ADJUSTABLE IN 1/2" INCREMENTS.
- 6. SHELF LADDERS, PERFORATIONS IN WALL, NOT SHOWN.
- 7. NYLON ROLLER DOOR CATCH, SPRING LOADED, NOT SHOWN.
- 8. DOOR KNOB.
- 9. LOCK, CYLINDER, CHANGEABLE CORE, KEY REMOVABLE IN LOCKED POSITION ONLY. DOORS KEYED DIFFERENT.
- 10. 7/8" DIA. K.O. FOR 120 VOLT ELECTRICAL CONNECTION.
- 11. SWITCH FOR REMOTE WARNING LIGHT, OPTIONAL.
- 12. REMOTE WARNING LIGHT, NOT SHOWN, OPTIONAL.

NOTE: ONLY ITEMS WITH MARKED BOXES ARE INCLUDED.

**CARR**

Carr Narcotics Locker  
Installation, Operation & Maintenance  
Manual No. 371 Revised 4/13

Model NL20

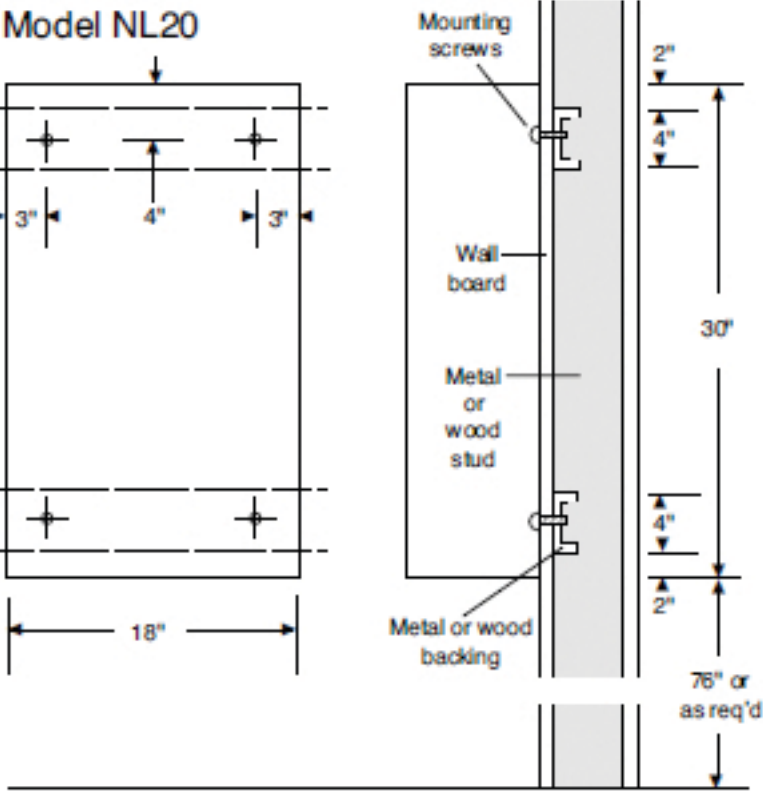

Installation

1. Check local codes before beginning installation.
2. Drill holes in back of cabinet in locations shown to attach to backing. If backing is not used and instead you choose to screw directly into studs (if possible), drill holes in back of cabinet at appropriate locations.
3. Place cabinet on wall and secure with screws appropriate for type of wall material.
4. Check that cabinet is level.

Operation

Cabinet can be unlocked using provided keys. Keys cannot be removed in the unlocked position. If key is lost or locks need to be changed, contact Carr.

Maintenance

The cabinet is maintenance free. To clean, use a mild detergent with water and a soft cloth. Wiping cabinet with a mineral oil on a soft cloth will make the stainless steel shine.

**CARR**