

ENGLISH

What do I need to know about the infant/child pads cartridge?

- The infant/child pads cartridge is designed for use with the HeartStart HSI Defibrillators. When used with the infant/child pads cartridge, the defibrillator delivers a lower-energy shock, and the optional CPR Coaching is tailored for use on a child.
- To keep the pads from drying out, keep the infant/child pads cartridge in its unopened foil pouch. Store it near your defibrillator, until you need to use the pads. It is recommended that you store the HeartStart Defibrillator with an adult pads cartridge installed.
- Be sure to note the expiration date of the pads cartridge when you receive it.
- CPR for infants and children is different than for adults. It is important to get training in infant/child CPR.
- If you have concerns about your child's health, be sure to talk to your child's doctor.

How do I know when to use the infant/child pads cartridge?

- Use the infant/child pads on an infant or a child less than 8 years old or weighing less than 55 pounds (25 kg) who may be in sudden cardiac arrest (SCA).
- If you are not sure of the child's exact age and weight, or if infant/child pads are not available, do not delay treatment:
- Use adult pads but place one pad on the chest and one in the center of the back, as shown in Emergency Instructions, below.
- A victim of sudden cardiac arrest does not respond when shaken, and is not breathing normally. If in doubt, apply the pads.

How do I install the infant/child pads cartridge?

- To install the infant/child pads cartridge, remove the adult pads cartridge from the defibrillator. Slide the latch at the top of the defibrillator to release the cartridge ❶. Lift out the adult pads cartridge.
- Open the foil pouch and take out the infant/child pads cartridge ❷.
- With the PULL handle side up, put the lower end of the cartridge into the pads compartment ❸.
- Press down the top end until the cartridge clicks into place with the PULL handle pressed down ❹.




- After using the infant/child pads, remove the cartridge. Install an adult pads cartridge before storing the defibrillator.

PHILIPS MEDICAL SYSTEMS

EMERGENCY INSTRUCTIONS

- Infants and children more often become unresponsive due to breathing or other issues than to heart problems. When responding to cardiac arrest in an infant or child, provide infant/child CPR while a bystander calls EMS and brings the HeartStart. If no bystander is available, provide 1-2 minutes of CPR before calling EMS and retrieving the HeartStart. If you witnessed the child's collapse, call EMS immediately and then get the HeartStart.
- Insert the infant/child pads cartridge and turn on the HeartStart Defibrillator. It will give you voice instructions for each step.
- Pads placement for infants and for children under 8 years old or 55 pounds (25kg) is different from adult pads placement. Place one pad in the center of the chest, between the nipples. Place the other pad in the center of the back.

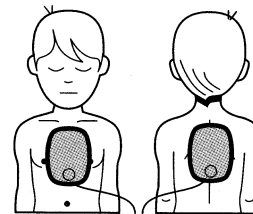

PHILIPS MEDICAL SYSTEMS