non-medical device – use for general wellness only

IMPORTANT: READ ALL WARNINGS & CAUTIONARY STATEMENTS TO REDUCE THE RISK OF INJURY WHEN USING THIS PRODUCT. PLEASE KEEP THE INSTRUCTIONS AND ADDITIONAL INFORMATION IN A SAFE PLACE. DO NOT LEND OUT OR GIVE THE PRODUCT TO A THIRD PARTY WITHOUT THESE INSTRUCTIONS.

The SISSEL® neck pillow is an ergonomic pillow that supports the cervical spine due to its special shape and thus relieves the entire shoulder-neck area.

The SISSEL® Soft Plus and the SISSEL® Classic Plus are height-variable by an insert to adjust the neck pillow individually.

The special shape positions the cervical spine in an anatomically correct, physiological position and relieves the neck muscles and intervertebral discs. In this way, the SISSEL® neck pillow makes a significant contribution to relaxing the neck muscles and thus promoting deep, healthy, restful sleep.

Lie on your side with your head on the SISSEL® neck pillow so that the high side is against your shoulder and the cervical spine is supported by the pad. The SISSEL® neck pillow is correctly positioned when the high side fills the area where the cervical spine meets the shoulder.

Safety Instructions:

⚠ WARNING

- DO NOT use the product if you suffer from any disorders such as acute injuries and/or sprains of the cervical spine, acute cervical spine syndrome (cervical syndrome), acute cervical spine vertigo, acute radiating pain, acute or chronic nerve root inflammation of the cervical spine, etc.
- STOP USING if you feel any pain.

⚠ CAUTION

- In case of uncertainty or existing health problems in the Shoulder neck area or cervical spine, use the product only after consulting a doctor.
- Do not store in extreme hot or cold environments – such as vehicles, outdoor sheds, etc.

NOTICE

- When using the SISSEL® neck pillow, the new sleeping position may initially cause a change in sleeping habits for a few nights. As a rule, this adjustment takes 2 to 3 days. Only in exceptional cases can it take longer.
- Regularly check the integrity of the product
- Do not use the product if it is damaged. Dispose damaged products correctly and never repair them.
- Keep the product away from radiators or other heat sources and avoid direct sunlight.
- Keep the area of use free of sharp or pointy objects.
- The core can be hand-washed at lukewarm temperature using a mild detergent. Carefully squeeze out by hand, never wring out! Then leave to air dry on a grid (place a towel underneath) for a sufficiently long time (approx. 1 week). DO NOT dry in the sun or on the heater.
- The velour cover can be removed and is machine washable at 40°C (104°F). Then air dry.
- Always refer to the instructions given on the care label.

Important information, please note:

This product is suitable for repeated use.

Discoloration of the pillow core is possible and does not affect the function of the product.

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