IMPORTANT: READ ALL WARNINGS & CAUTIONARY STATEMENTS TO REDUCE THE RISK OF INJURY WHEN USING THIS PRODUCT. PLEASE KEEP THE INSTRUCTIONS AND ADDITIONAL INFORMATION IN A SAFE PLACE. DO NOT LEND OUT OR GIVE THE PRODUCT TO A THIRD PARTY WITHOUT THESE INSTRUCTIONS.

The SISSEL® Press Egg is a training tool for hand and forearm.

The product can be used for targeted building of muscles and improving mobility in hand and forearm, as well as for mobilization of fingers and hand.

The SISSEL® Press Ball comes with exercise instructions.

Colors (strength):

- pink (soft)
- blue (medium)
- green (strong)
- orange (extra strong)

Safety Instructions:

⚠ WARNING

- Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- DO NOT use the product if you find any defects. Dispose damaged product correctly and never repair it.
- STOP USING immediately if you feel uncomfortable or any pain or dizziness.
- The product is NOT intended for consumption or absorption into the body.
- Keep the exercise area free of sharp objects and obstructions.
- This product is not a toy, keep it out of reach of children.
- Never allow pets near the product.

⚠ CAUTION

- Before beginning your exercise program, please see a healthcare provider or physician for appropriate exercise advice and safety precautions.
- If you feel uncomfortable during the exercises, stop using and consult a physician.
- Always practice your exercises carefully, follow the exercise instructions exactly and adapt the exercises to your fitness level.
- DO NOT kick or throw.

NOTICE

- To clean the product, wipe with a damp cloth. Then rub dry. If necessary, use suitable wipe disinfectant and check instructions of disinfectant first. If necessary, powder with commercially available baby powder.
- Storage: Protect product from direct sunlight and store in a cool, clean and dry place.

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