IMPORTANT: READ ALL WARNINGS & CAUTIONARY STATEMENTS TO REDUCE THE RISK OF INJURY WHEN USING THIS PRODUCT. PLEASE KEEP THE INSTRUCTIONS AND ADDITIONAL INFORMATION IN A SAFE PLACE. DO NOT LEND OUT OR GIVE THE PRODUCT TO A THIRD PARTY WITHOUT THESE INSTRUCTIONS.

The SISSEL® Neck Relax combines relaxation and massage.

Thanks to the ergonomic fit, you can comfortably place your neck on the SISSEL® Neck Relax and relax from the very first second. The semi-rigid nubs effectively massage the neck muscles. In addition to the muscles, the fasciae in the neck area are also worked on, which can loosen additional tension.

With the SISSEL® Neck Relax you can not only relax the neck, but also effectively mobilize your thoracic vertebrae.

The SISSEL® Neck Relax can be used for other parts of the body, too.

Such as a relaxing foot massage or a headrest during Pilates.

The product is air-filled, and its material is 100 % PVC.

The dimensions of the product are approx. 16,5 cm (6,5") x 14,5 cm (5,7") x 6,5 cm (2,6").

Safety Instructions:

⚠ WARNING

- Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed.
- STOP USING immediately if you feel uncomfortable or any pain or dizziness.
- DO NOT use the product if you suffer from any disorders.
- DO NOT stand on the product. Sit down when using the product for foot massage.
- Keep the exercise area free of sharp and pointed edges (e. g. keys, scissors, pens, etc.).
- DO NOT kick or throw the product.

⚠ CAUTION

- Remove all objects from your body, clothes and pockets that could potentially puncture the product during use and afterwards when stored.
- In case of uncertainty or existing health problems, use the product only after consulting a doctor.

- Do not store in extreme hot or cold environments – such as vehicles, outdoor sheds, etc.
- This product is not a toy, keep it out of reach of children.
- Never allow pets near the product.

NOTICE

- Regularly check the integrity of the product.
- Do not use the product if it is damaged. Dispose damaged products correctly and never repair them.
- Keep the product away from radiators or other heat sources and avoid direct sunlight.
- Keep the area of use free of sharp or pointy objects.
- To clean the product, wipe with a damp cloth. Then rub dry. If necessary, use wipe disinfectant suitable for PVC surfaces and check instructions of disinfectant first.
- Storage: Protect product from direct sunlight and store in a cool, clean and dry place.

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