IMPORTANT: READ ALL WARNINGS & CAUTIONARY STATEMENTS TO REDUCE THE RISK OF INJURY WHEN USING THIS PRODUCT. PLEASE KEEP THE INSTRUCTIONS AND ADDITIONAL INFORMATION IN A SAFE PLACE. DO NOT LEND OUT OR GIVE THE PRODUCT TO A THIRD PARTY WITHOUT THESE INSTRUCTIONS.

The SPINEFITTER by SISSEL® Linum is a hot/cold pack filled with linseed for targeted, local heat or cold treatment of the muscles in the spinal column. Warming and cooling treatments use the reactions of the skin, the subcutis, the musculature and deep tissue layers to the respective temperature stimulus.

The SPINEFITTER by SISSEL® Linum can be used individually or in combination with the SPINEFITTER by SISSEL®, on which it can be securely attached thanks to the side pockets and the central band roles.

Warming

Place the product folded in the microwave and try to find the optimum heating time (e. g. 600 Watt: 2 min; 1000 Watt: 60-90 sec).

Remove the product from the microwave and check the temperature. If further heating is required, shake the product, and heat it again for 20 seconds. Repeat this until the desired temperature is reached.

Preheat the oven to 70°C (160°F). Place the product folded on the middle shelf of the oven for 4 to 6 min. Remove the product from the oven and check the temperature. If further heating is required, shake the product, and heat it again for 1 to 2 min. Repeat this until the desired temperature is reached.

For a deep, moist heat, add a fireproof, waterfilled bowl in the microwave or in the oven.

Cooling

Place the product in a suitable plastic bag in the refrigerator or freezer for at least 2 hours.

The SPINEFITTER by SISSEL® Linum is designed for a maximum user weight limit of 150 kg (330 lbs).

Safety Instructions:

⚠ WARNING

- Check the condition of the product before each use. If damaged, the function of the product and safety are no longer guaranteed. Leaking filling can cause skin burns and risk of choking.
- Risk of injury! The heat may be unevenly distributed at first. It is important to shake the product well after each heating cycle to distribute the temperature evenly in the filling.
- Risk of injury! Never place the product directly on the skin, always use a thin cotton cloth or your cloth as an intermediate layer.
- Check the temperature at your wrist before use. Pay extra attention before use by heat/cold-sensitive people, such as children under the age of 3.
- Use on children only under the supervision of and with instructions from adults. DO NOT use on infants or children under the age of three! They are not able to remove a source of heat/cold if they feel uncomfortable.
- DO NOT use the heated product if you suffer from any disorders like inflammation, open wounds, oedema, infections, dermatitis, thrombosis etc.
- DO NOT use the cooled product if you suffer from any disorders like cold intolerance/allergy, Raynaud's syndrome, open wounds, etc.
- STOP USING immediately if you feel uncomfortable or any pain or dizziness occurs.
- The product and their accessories are no toys and should be kept out of the reach of children. DO NOT allow children to play with them. Danger of choking!

⚠ CAUTION

- Only apply the product for as long as you find it comfortable - max. 20 minutes. Before applying the grain pillow again, the treated area should remain uncovered for 20 minutes.
- DO NOT use the product in combination with the SPINEFITTER by SISSEL® for exercises performed in a standing or kneeling position, as slipping of the product could lead to loss of balance or slipping off the exercise equipment.
- Do not wet the pack.
- This product is not a toy, keep it out of reach of children.
- The filling of the product is not suitable for consumption.

NOTICE

- Supervise warm up. Overheating can destroy the product and there is a risk of fire. DO NOT heat the product in a water bath or grill.

Care & Hygiene:

The product cannot be washed, cleaned or disinfected. Always protect from moisture, dry carefully.

DO NOT use detergents, cleaning agents and disinfectants.

For storage, protect the product from direct sunlight and store it in a dark, dry place.

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