

NC475 Tek-CARE Appliance Server IL1011 Specification Sheet

Tek-CARE Nurse Call and Tek-ALERT Alert Integration Systems • Rev. 15 - 6/2025

Features

- † Proprietary nurse call appliance server running the Tek-CARE OS. The Tek-CARE Appliance Server is available in two different variants:
 - † The NC475 is a headless appliance server
 - † The NC475DESK is supplied with a 24" touch-screen monitor (size may vary) and wireless keyboard and mouse
- † Functions as a tone-visual master station for the Tek-CARE500 and Tek-CARE570 systems
- † Includes paging software module for automatic and manual pages to staff
- † Software module options include event monitoring, reporting, Staff App, Apple TV displays, email output, and more
- † Two network interface ports provide connections to the Tek-CARE Network and facility LAN
- † Six RS232 serial ports provide connections for a paging transmitter, a central monitoring interface, and Tek-CARE500 and Tek-CARE570 Receivers

Description

The NC475 Tek-CARE Appliance Server provides the platform to install and run various software modules for Tek-CARE communications platform. The Tek-CARE Appliance Server is designed to be desk mounted or installed in a rack mount application. Rack mount ears and mounting screws are included and when rack mounted, uses 2U of rack space.

The Tek-CARE Appliance Server also supplies ports for the Tek-CARE Network and the facility's LAN, the radio pocket paging system's paging transmitter, Tek-CARE 300III Integration, the NC570, NC510UL, NC510ES, and NC510 FA Receivers.

The LCD screen on the Tek-CARE Appliance Server displays any calls and faults active on the nurse call system. The LED indicators to the left of the LCD provide power, fault, and call status at a glance. A simple configuration menu is also accessible from the front display.

The NC475DESK can function as a tone-visual master for the Tek-CARE platform. Calls, faults, and basic system configuration options are available, but audio communication is not available from the NC475 or NC475DESK. For audio communication, purchase the appropriate master station for your Tek-CARE system.



NC475DESK

Specifications

Dimensions: NC475:2.5"H×13.45"W×7.875"D
(64 mm × 342 mm × 199 mm)

Monitor: 14"H × 21.3"W × 1.9"D
(356 mm × 541 mm × 48 mm)

Power: AC/DC Power Adapter included
Input: 100-240VAC, 2A, 50/60Hz
Output: 12VDC, 5A, 60W Max
Battery Backup: TekTone PK250B or equivalent required

Temperature: 0-40°C

Humidity: 0-95% (non-condensing)

Applicable Systems

Tek-CARE®120 Nurse Call System Tek-CARE®300III Nurse Call System Tek-CARE®400 P5/P5+ Nurse Call Systems Tek-CARE®400 GEN3 Nurse Call System Tek-CARE®500 Wireless Nurse Call System Tek-CARE®570 Wireless Nurse Call System Tek-CARE®600 Alert Integration Manager Tek-CARE®700 Wander Management System Tek-CARE®3000 Nurse Call System

Listings



UL® 1069 Listed and cUL® Listed to CSA C22.2 No. 205; UL® File Number E71599.

Software License Selections

Individually available software modules run on the Tek-CARE Appliance Server, adding ancillary features to the healthcare communications system.

System Connectivity:

- **LS120 Tek-CARE120 connectivity license** enables the Tek-CARE Appliance Server to import events from Tek-CARE120 Nurse Call Systems.
- **LS301 Tek-CARE300II & LS300 Tek-CARE300III connectivity license** enables the Tek-CARE Appliance Server to import events.
- **LS331 Tek-CARE3000 connectivity license** enables the Tek-CARE Appliance Server to import events from Tek-CARE3000 Nurse Call Systems.
- **LS400 Tek-CARE400 connectivity license** enables the Tek-CARE Appliance Server to import events from Tek-CARE400 Nurse Call Systems.
- **LS435 Tek-CARE400 GEN3 connectivity license** enables the Tek-CARE Appliance Server to import events from Tek-CARE400 GEN3 Nurse Call Systems.
- **LS460 Tek-CARE Media Gateway connectivity license** provides integration between the Tek-CARE Network and VoIP telephone systems. This includes 4 audio paths. LS465 adds 4 additional audio paths.
- **LS500 Tek-CARE500 connectivity license** enables the Tek-CARE Appliance Server to import events from FA, ES, and UL version Tek-CARE500 Wireless Nurse Call Systems.
- **LS570 Tek-CAREconnectivity license** enables the Tek-CARE Appliance Server to import events from the Tek-CARE570 Wireless Nurse Call System)
- **LS600 Tek-ALERT connectivity license** enables the Tek-CARE Appliance Server to import events from other systems, such as nurse call and fire alarm. (One LS600 required per interface.)
- **LS700K Tek-CARE700 connectivity license with Starter Kit** enables the Tek-CARE Appliance Server to import events from our wander management system. It includes the NC708 Starter Kit with ID-TAD, testing tag, #6 spanner hex bit, and programming adapter for door controllers.

Event Monitors & Reporting:

- **LS610-series Reporting** records all activity on the Tek-CARE Appliance Server, and provides custom reports on this activity for followup review and evaluation by management. Reports are viewed using a web browser by any computer networked to the Tek-CARE Appliance Server. They may also be printed, saved as HTML files for later viewing in a web browser, or saved as a CSV files for viewing in a spreadsheet program (such as Microsoft Excel).

When LS453 Email Output software is combined with LS610-series Reporting, reports can be sent automatically via email. Available in packages for 100 points (LS610/1), 500 points (LS610/5), 1000 points

(LS610/10), 2500 points (LS610/25), 5000 points (LS610/50), and 10000 points (LS610/100). A point is one Tek-CARE120 or Tek-CARE400 addressable device, Tek-CARE500 transmitter, Tek-ALERT device ID, or Tek-CARE700 tag.

- **LS623-series Event Monitor for Windows** displays activity from all connected systems. This software is installed on the Tek-CARE Appliance Server, plus up to 255 personal computers networked to the Tek-CARE Appliance Server. (PCs must be running Microsoft Windows.) Available in packages of one license (LS623/1), five licenses (LS623/5), 10 licenses (LS623/10), or 25 licenses (LS623/25).

ADT Interface:

- **LS452 ADT Interface** receives Admit, Discharge and Transfer information from the facility's hospital information system (HIS)—including patient name, room number, bed number and doctor name—and inserts it into the corresponding fields in the Tek-CARE system's patient database. Information in these fields is then visible to facility staff on touchscreen master stations.

Email Output:

- **LS453 Email Output** enables email addresses to be added as staff devices. The software sends calls and other events via email to the assigned staff members. Events can also be sent to anything that has an email gateway—for example, sending an SMS message to a cell phone.

Tek-CARE Staff App:

- **LS620/LS630 Tek-CARE non-voice/voice App Server license** enables the Tek-CARE Appliance Server to send events via the facility's Wi-Fi to mobile devices running the LS621/LS631-series app.
- **LS621/LS631-series Tek-CARE non-voice/voice Staff App** enables mobile iOS and Android devices to act as portable event monitors. Up to 255 iOS and/or Android devices running the Tek-CARE app can be active on the system at one time. Install licenses on the Tek-CARE Appliance Server, and download the app from the Apple Store. See the IL1020 specification for device requirements. Available in packages of one license (LS621/1 or LS631/1), five licenses (LS621/5 or LS631/5), 10 licenses (LS621/10 or LS631/10), 25 licenses (LS621/25 or LS631/25), 50 licenses (LS621/50 or LS631/50), 100 licenses (LS621/100 or LS631/100), and 200 licenses (LS621/200 or LS631/200).

Tek-CARE TVs:

- **LS622-series Tek-CARE TV for the Apple TV App licenses** enables a 4th Generation (or newer) Apple TV with an HDMI connected widescreen TV or monitor to act as an event monitor. Up to 10 Apple TVs can be active on the system at one time. Install licenses on the Tek-CARE Appliance Server. Available in packages of one license (LS622/1), five licenses (LS622/5), 10 licenses (LS622/10), 25 licenses (LS622/25), 100 licenses (LS622/100), and 200 licenses (LS622/200).