

TekTone SF571 Universal Transmitter

Description

The SF571 is a wireless wall-mounted transmitter for use with any standard NO or NC contact or sensor. The transmitter has a hall effect sensor and magnet component to monitor doors or windows as well. Alarms are automatically

reset when returned to a normal state or when calls are acknowledged by staff, depending on system programming. The SF571 also provides for check-in and rounding capabilities that can be triggered by various reset signals available on the device.

The various device options are set via an internal Dip switch. Move the SW5 switch to OFF, make Dip switch changes, then move SW5 switch to ON to enable changes.



SF571
Universal Transmitter

Features

- Monitor auxiliary call devices
- 1 Monitor Normally Open or Normally Closed contacts
- 1 Supervised
- 1 Surface-mounted via screws or double-sided tape (provided)
- 1 Dip switch programming for various options
- 1 Built-in On/Off switch
- 1 Reset switch with On/Off option
- Facilitates Check-in and/or Rounding

Required Components

- Tek-CARE• 570 Nurse Call System
- NC570 Receiver
- NC571 Repeater
- NC475DESK

Technical Specifications

- **Dimensions:** 3.2"H₁ × 1.6"W × 0.75"D
(81 mm × 41 mm × 19 mm)
- **Construction:** Flame retardant ABS plastic
- **Distance:** 10 ft. maximum cable to contact
- **Battery:** 3.0V Lithium (BA007)
- **Environment:** 0° - 60°C
- **Frequency:** 433.92 MHz



UL Listing Information

- FCC Compliant.
- RoHS Compliant.