



Architects' and Engineers' Specifications

The bed station shall be a wireless, single gang bedside patient station, part number WNC-BS and shall include one IEA standard 1/4" jack for a single call cord, a "Call Placed" indicator, a "Call Cancel" button. When the call switch on the end of the call cord is activated, or if the call cord is removed from the jack, an alarm call will be registered at the Video Display Console, including the room number and name. The Video Display Console shall additionally display the arrival time, priority level and type of call. A light indication will also appear on the corridor lamp, corridor zone lamp and the station's "Call Placed" LED indicator. In new construction the station shall be housed in a Carlon A52171D deep 4/S plastic back box with single gang plastic mud ring or equivalent.

System Components

<u>Model No.</u>	<u>Description</u>
PC Server	Video Display Console PC
MV400	400 Console Master
MV200	200 Console Master
WNC-CC	Call Cord, 1/4" plug, 8 ft. length

WNC-BS Single Bedside Patient Station

The WNC-BS is a wireless bedside patient station equipped with one IEA standard 1/4" jack allowing the attachment of call cord. The station includes a "Call Placed" indicator which lights when a call is placed and a "Cancel" button to cancel the placed call. Both are mounted on a GE Lexan plastic single-gang panel. A call from a Bed Station activates a call information display at the Video Display Console, MV400 or MV200 Console. Other actions which will result in a display and light indication include the following: pressing the call cord button and removal of the call cord from the jack on the panel.

Specification

- **Dimensions:** Height: 4-1/2" (114mm)
Width: 2-3/4" (69mm)
Depth: 2-5/8" (66mm)
Weight: 7.2 ounces
- **Panel:** Flame retardant GE Lexan faceplate (UL flame rating 94V0)
- **Batteries:** Two CR123 Lithium batteries
- **Backbox:** Carlon A52171D deep 4/S plastic back box with single gang plastic mud ring
- **Surface Mount:** Wiremold 2344 single gang enclosure (2-3/4" deep)
- **Flush Mount:** Cooper BB10L single gang B-Line bracket

Features

- Simple push button calling
- Call placed indicator
- Call-reset push button
- Call automatically registers if call cord is removed
- Call is registered at the console with corresponding color indication and audio tone or voice.
- Call activates corridor lamp, corridor zone lamp, duty station, wireless LED reader board and other staff notification devices, if installed.